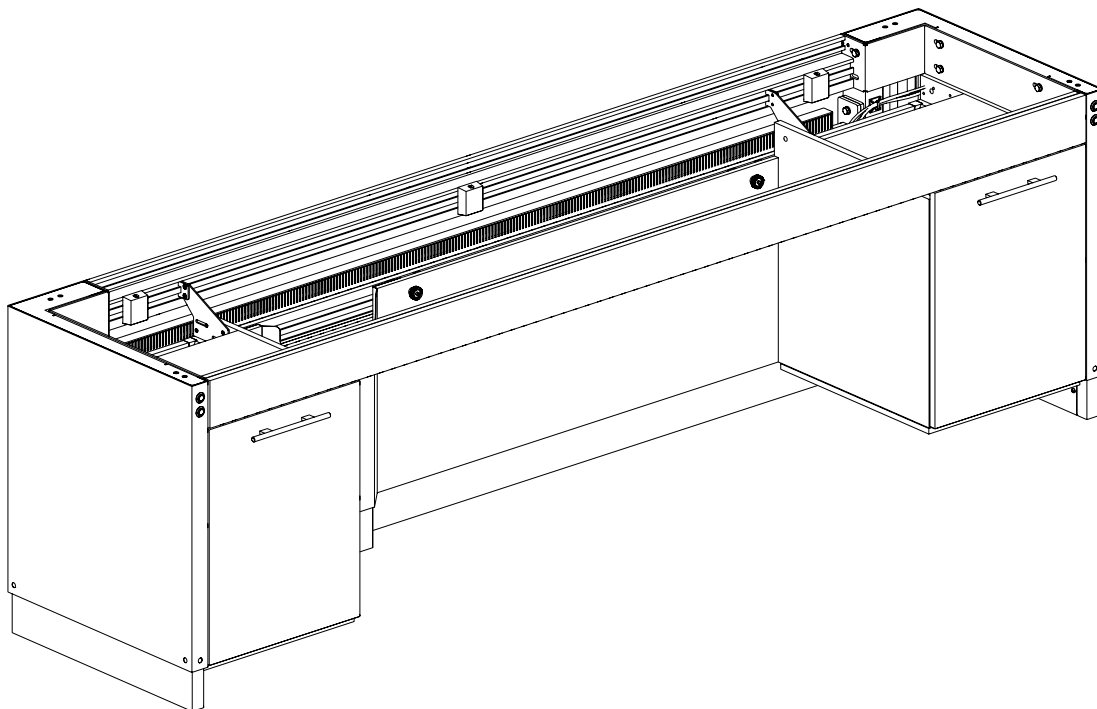
 **Installationsanvisning**
Sidelift 6400

 **Installation Instructions**
Sidelift 6400

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EN Preparations

Thank you for choosing to install a product from Granberg!

SideLift 6400 is an electrically height adjustable bench with a flexible system for suspended cabinets. The cabinets are placed where desired and can be temporarily removed / reassembled if conditions change. See examples of configurations on pages 32 and 33.

In order for the product to work safely and securely, it is essential that the installation instructions be followed. The installer should read and understand all the installation instructions before installation begins.

It is the installer who is responsible for floor structures meeting the set requirements and that fasteners can withstand the forces that may occur.

The installer is also responsible for the right authority and skills being available for electric and water works. After completed installation, the product shall be functionally tested according to sections on page 25.

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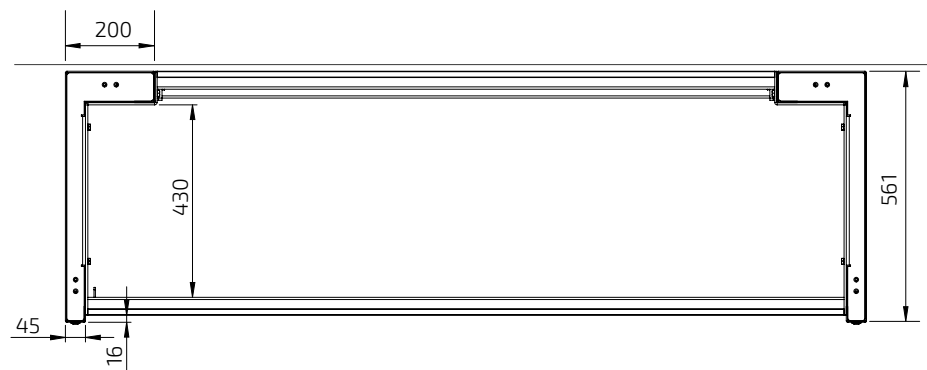
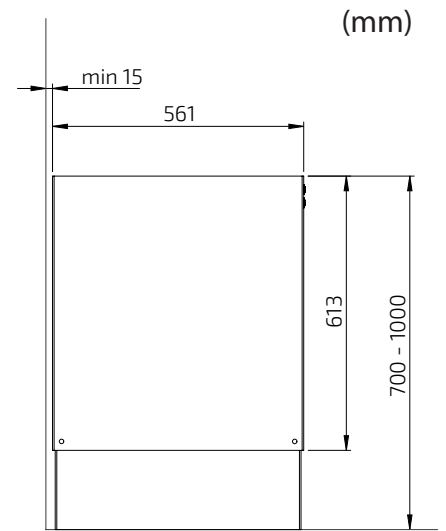
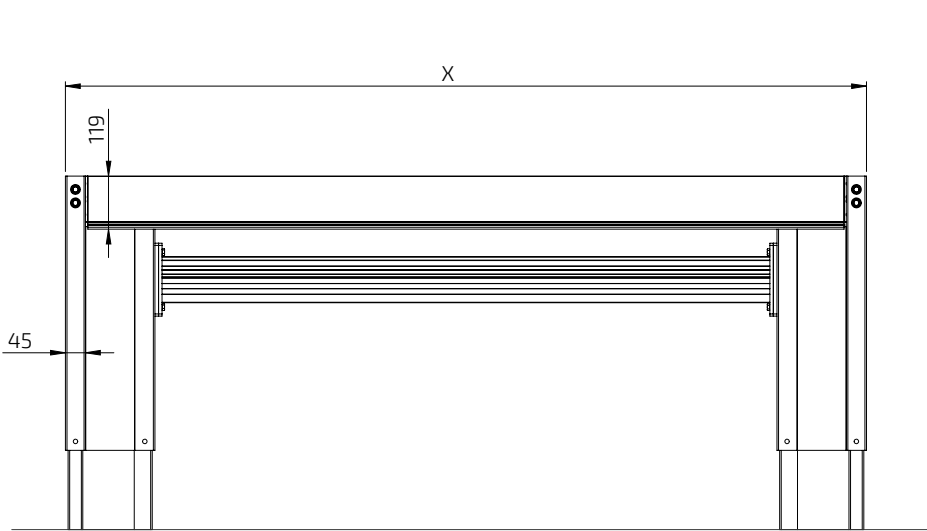
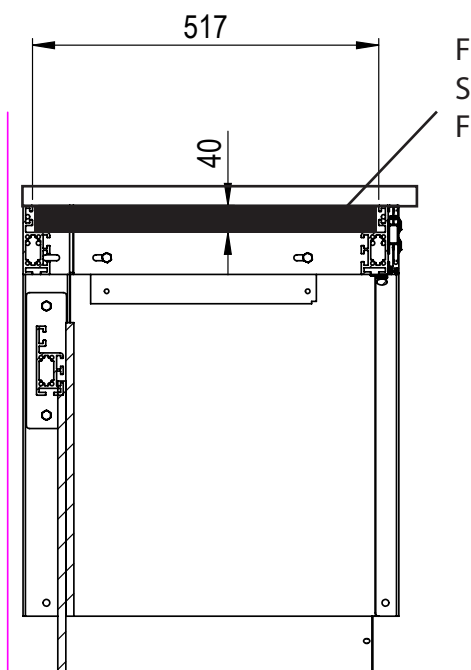
DE **Very Important!**

Check the stability of the floor before installation.

Maximum load on lifting unit is 400kg and dead load up to 100kg.

Ensure that the floor can withstand the concentrated load that occurs under motor units.

Make sure that the engine units can be fixed to the floor and that the floor structure is not porous.

In some cases, the floor may need to be reinforced and adapted in order for engine units to be adequately fixed.

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



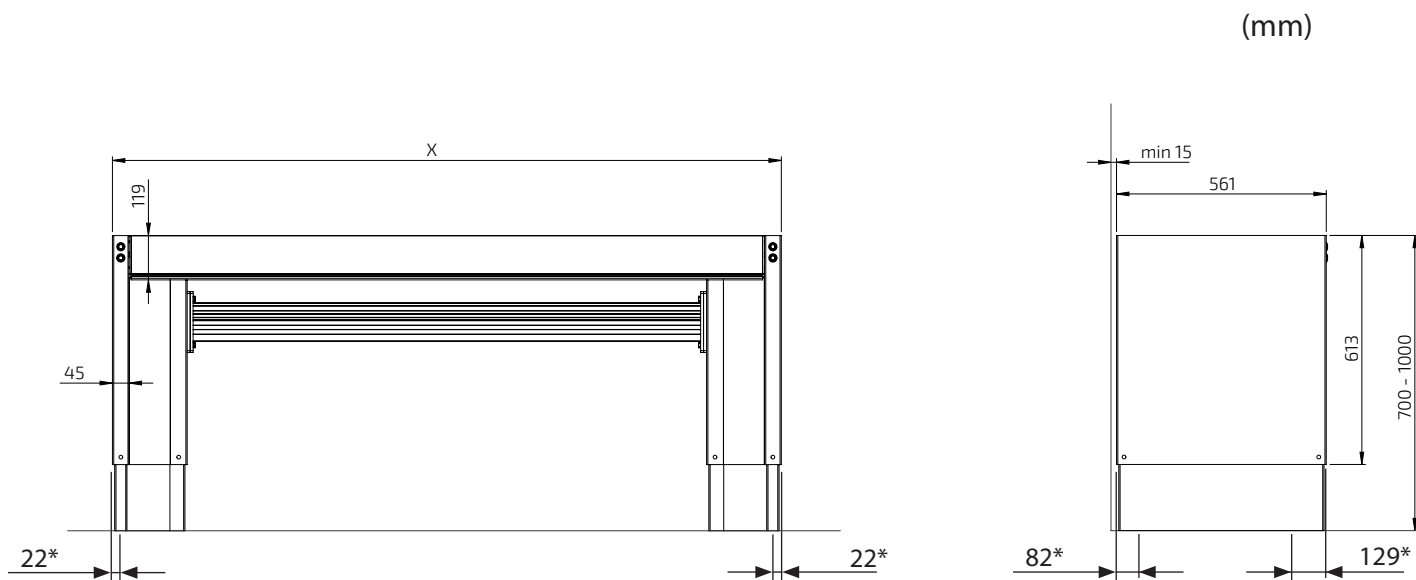
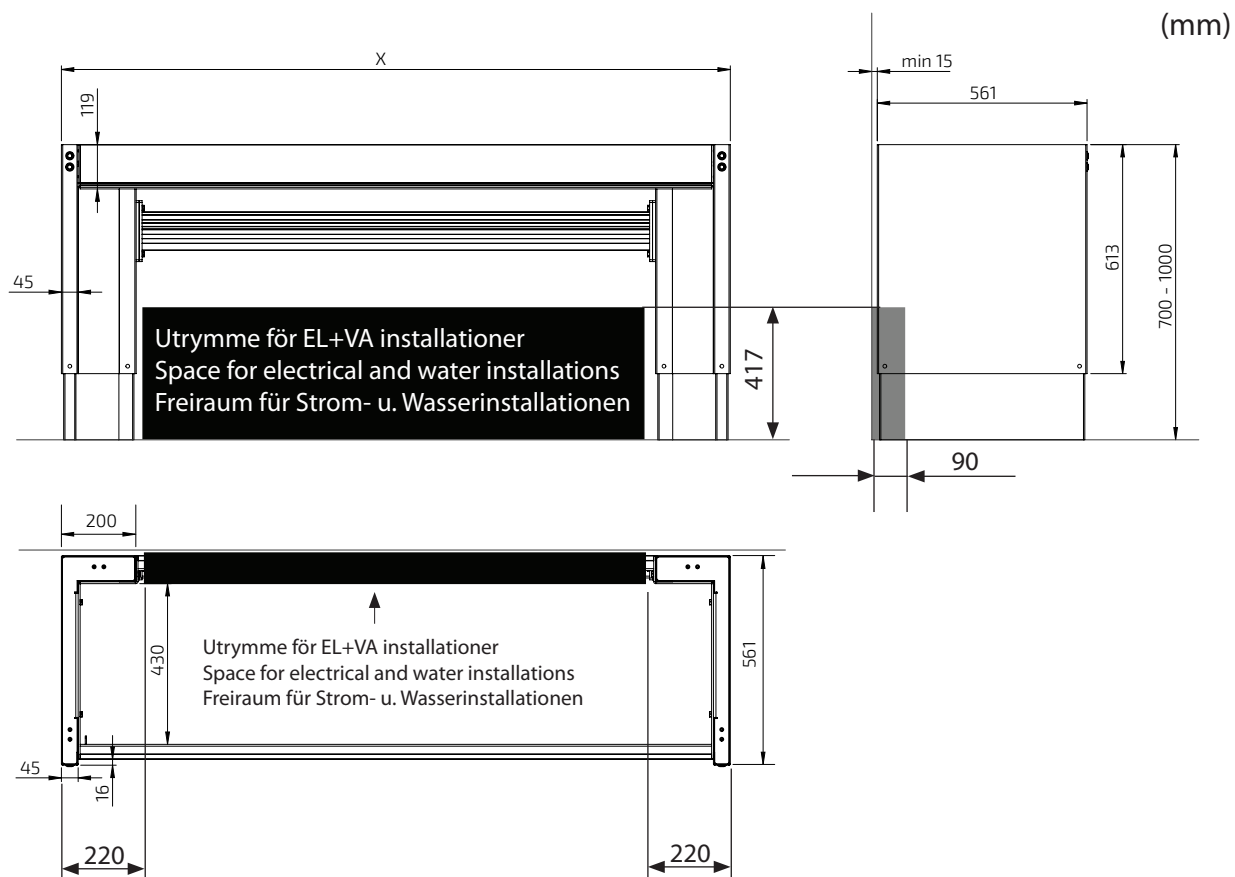
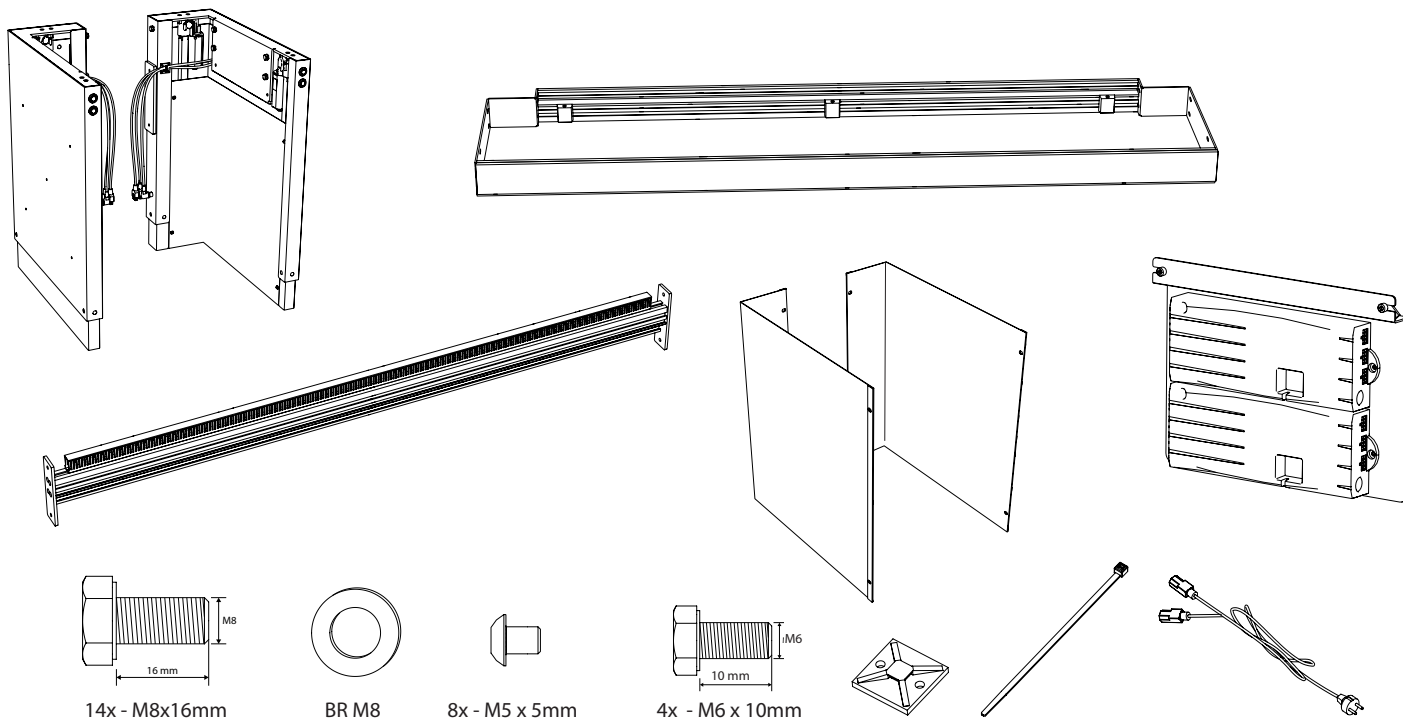
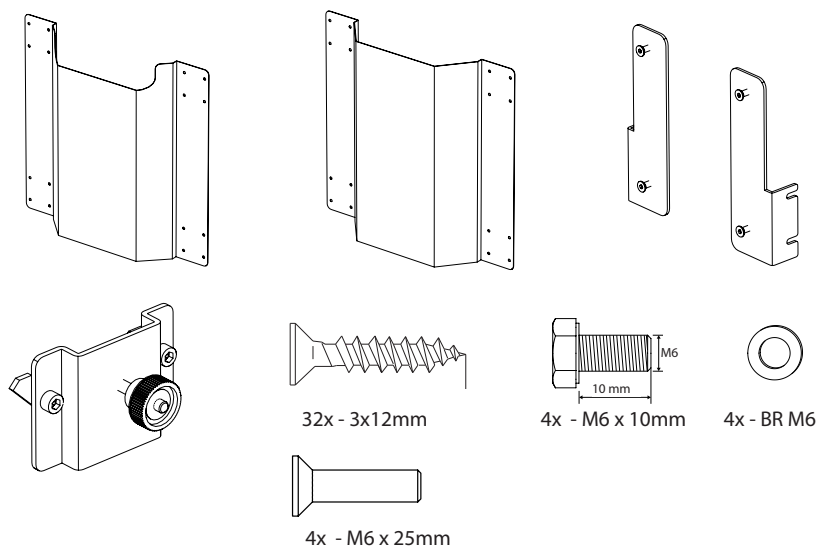
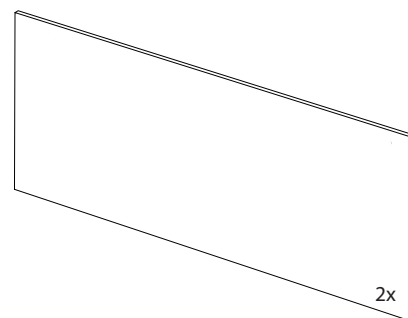
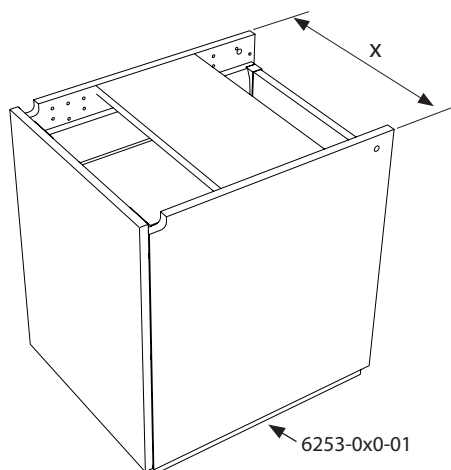
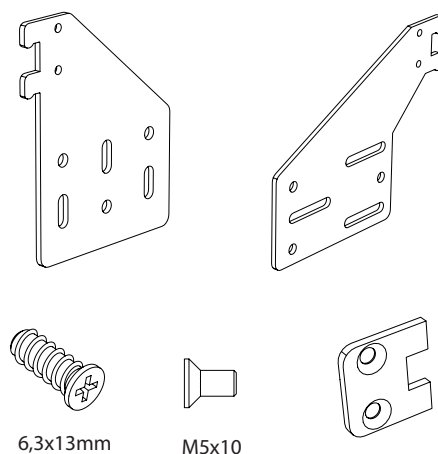
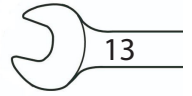
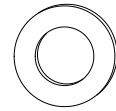
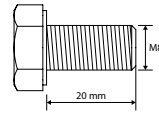
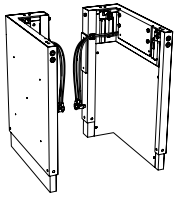
EN Place the motor units upside down on a flat surface on the floor and screw on the frame.
Align according to Figure 1.1.

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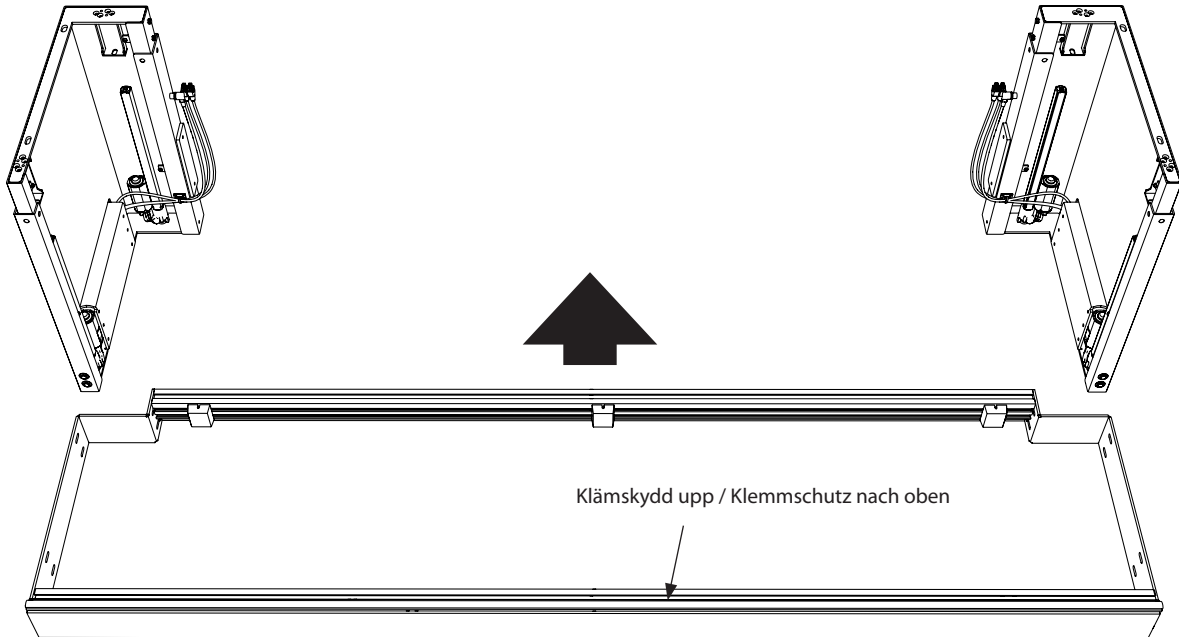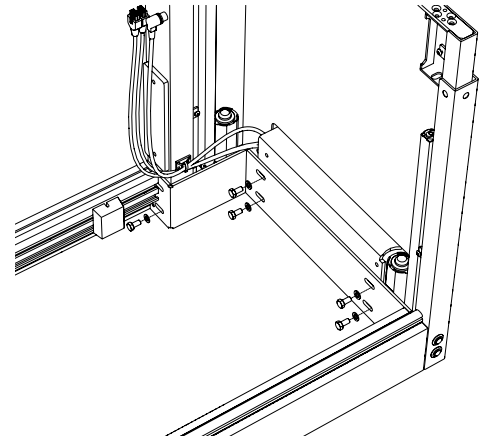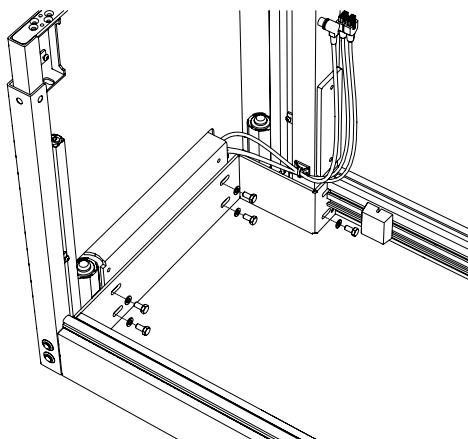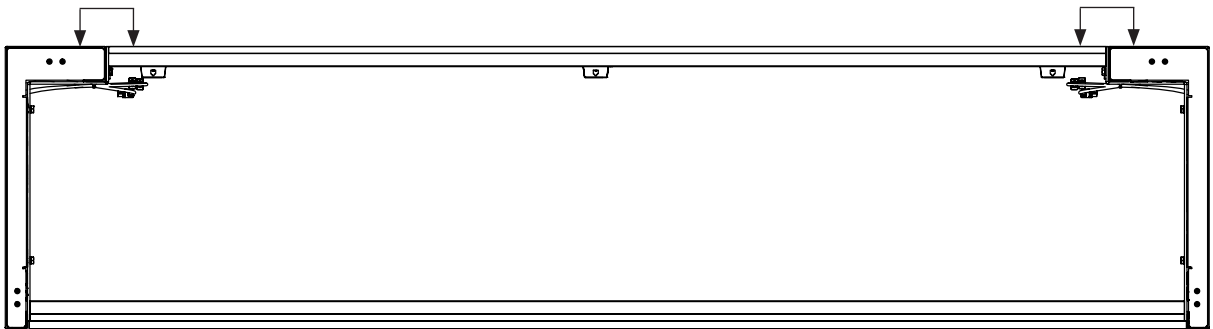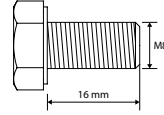
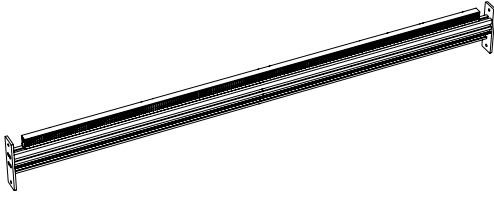
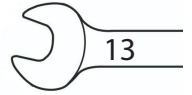
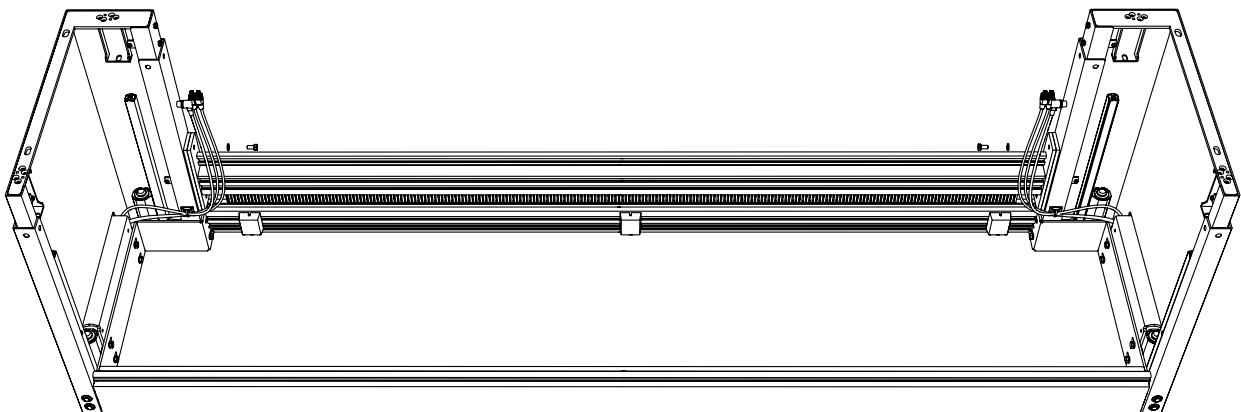
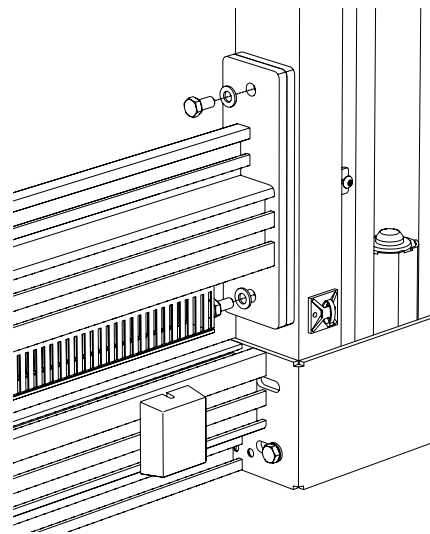
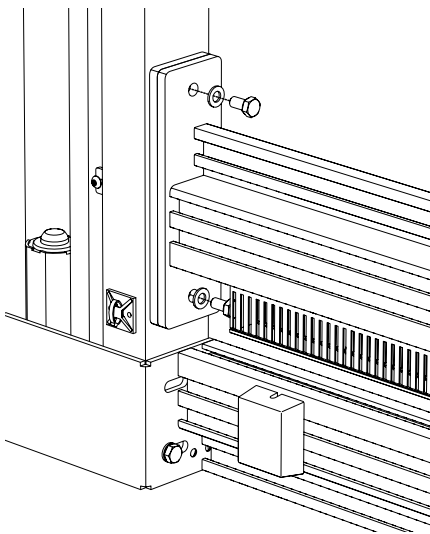
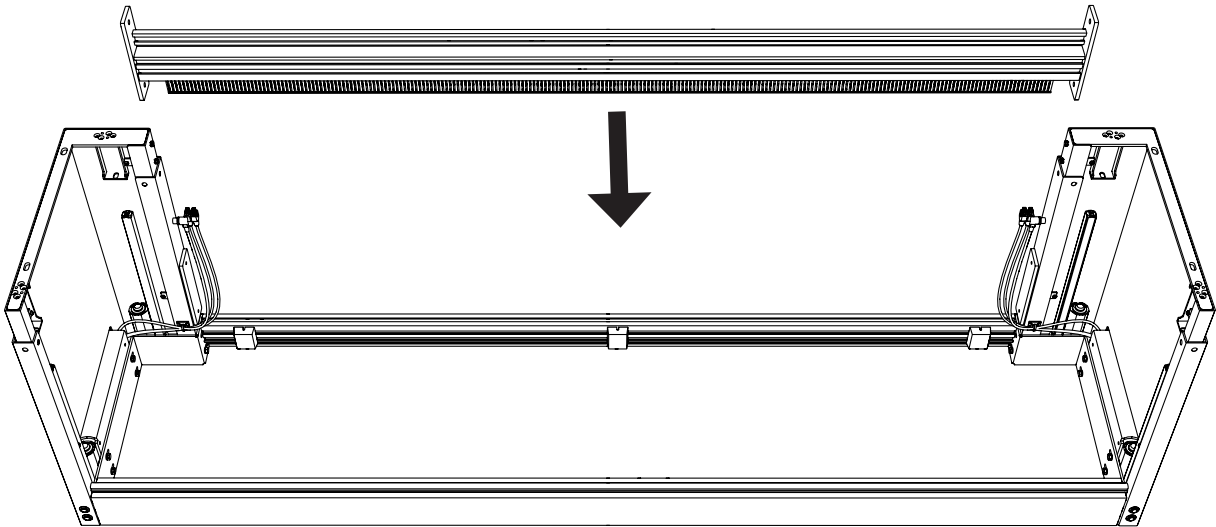
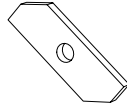
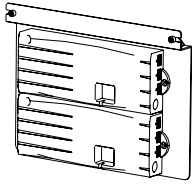
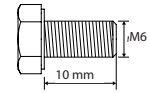
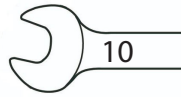
EN

Front view of lifting unit.

Screw the control panel onto the lower aluminium beam.

Move the control pane to the side to a position free from obstacles when raising 300mm.

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
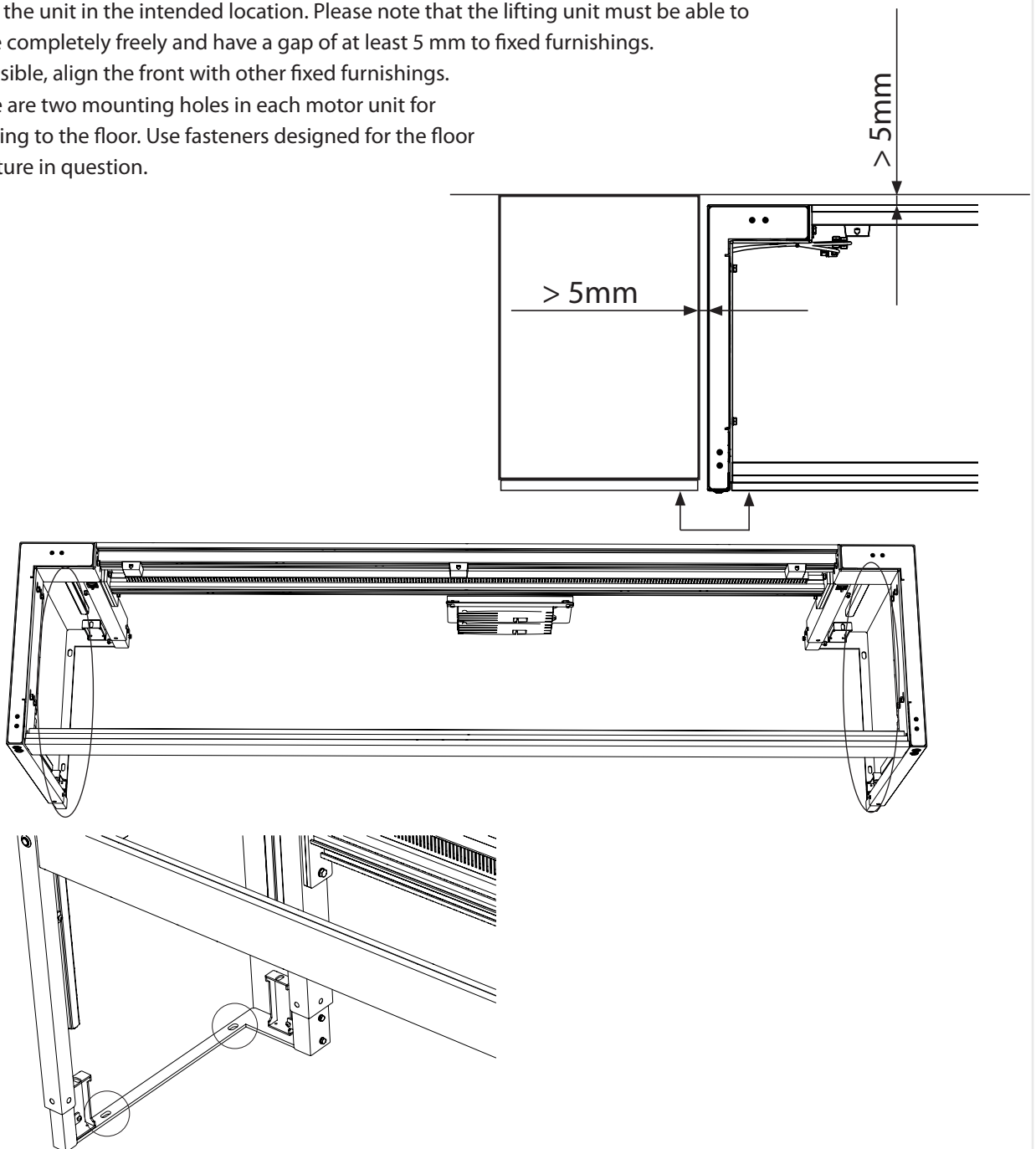
EN Place the unit in the intended location. Please note that the lifting unit must be able to move completely freely and have a gap of at least 5 mm to fixed furnishings.
If possible, align the front with other fixed furnishings.
There are two mounting holes in each motor unit for securing to the floor. Use fasteners designed for the floor structure in question.

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
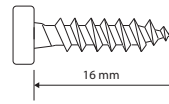
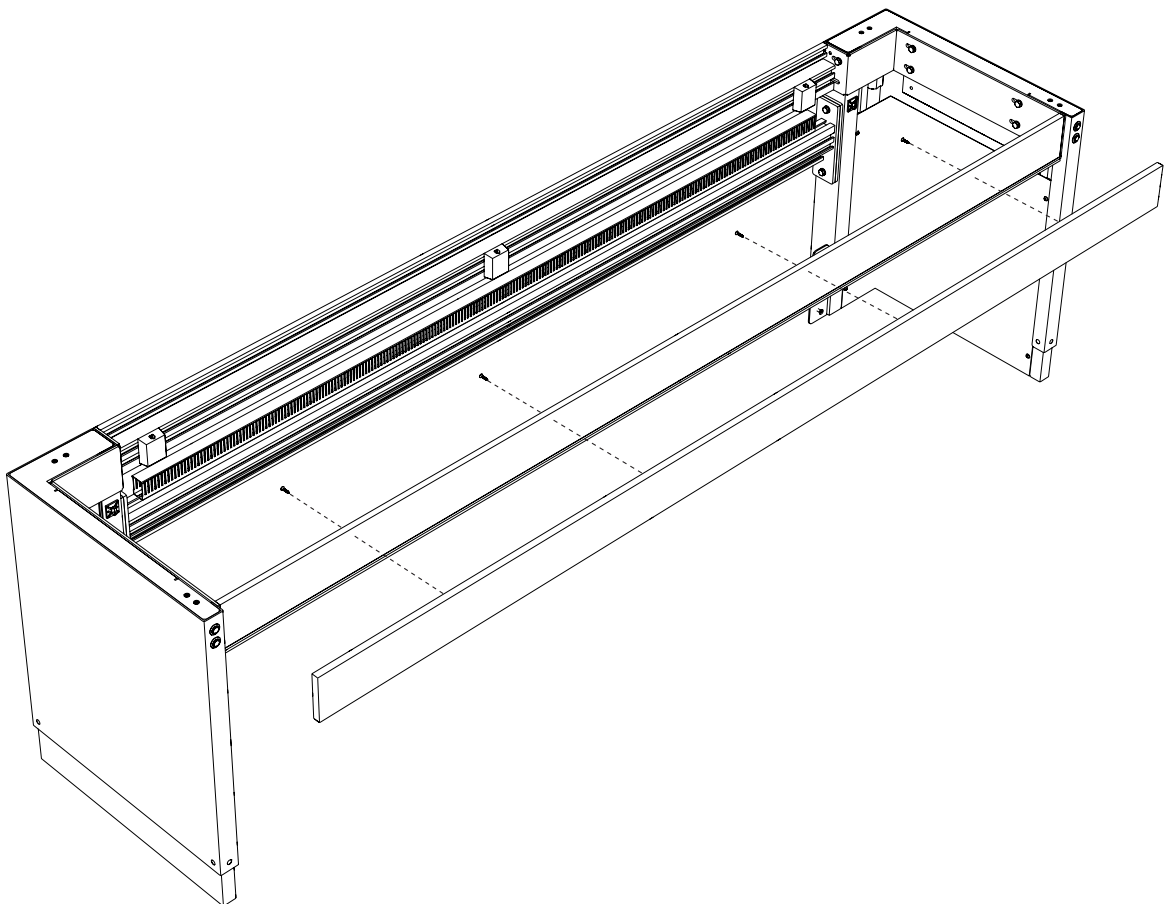
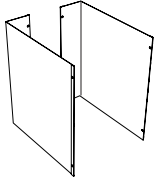
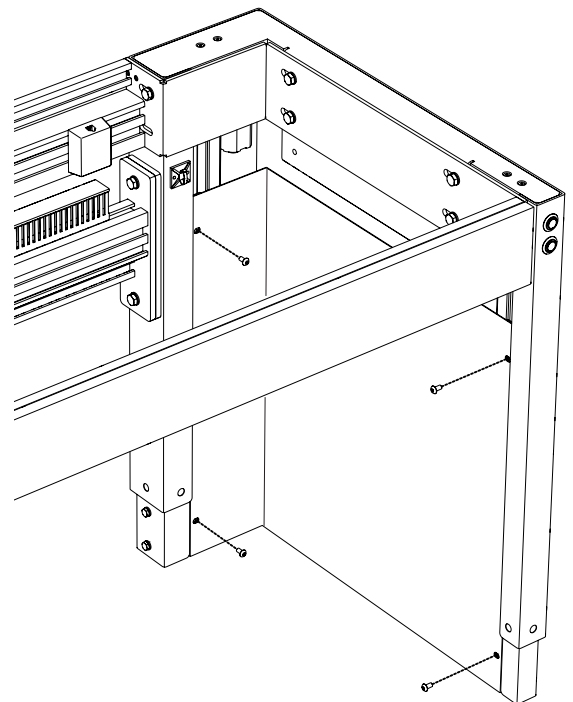
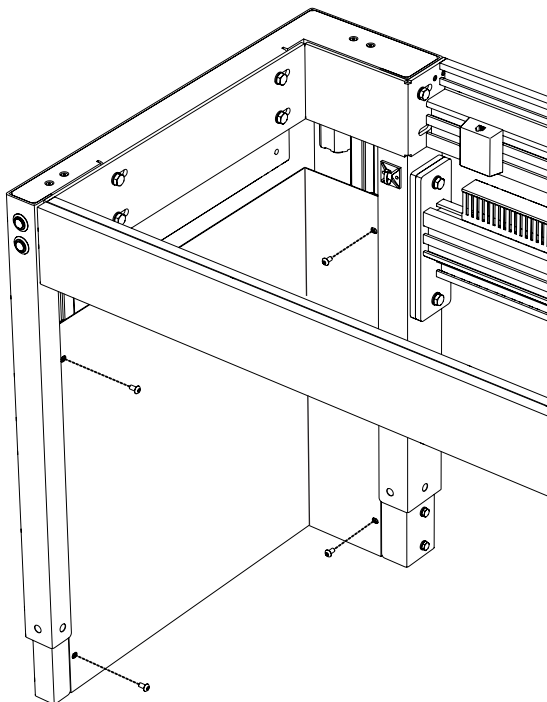
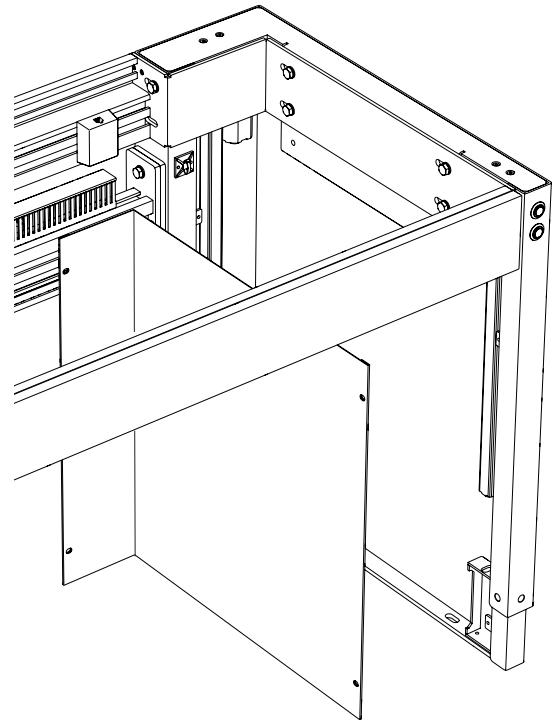
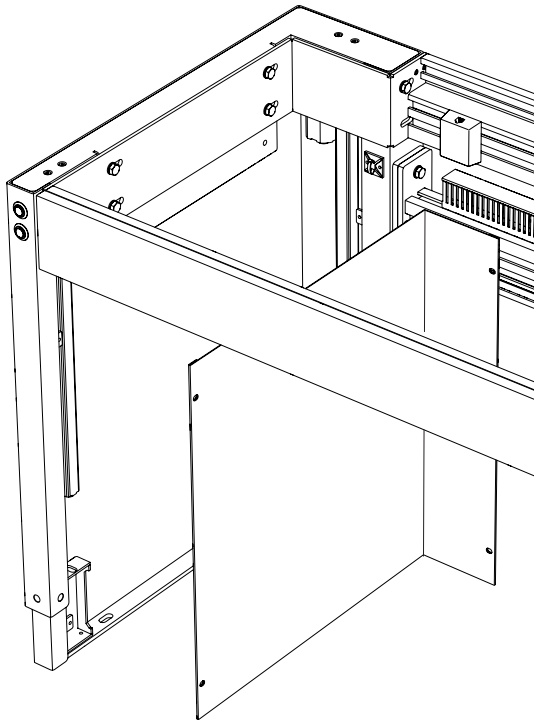
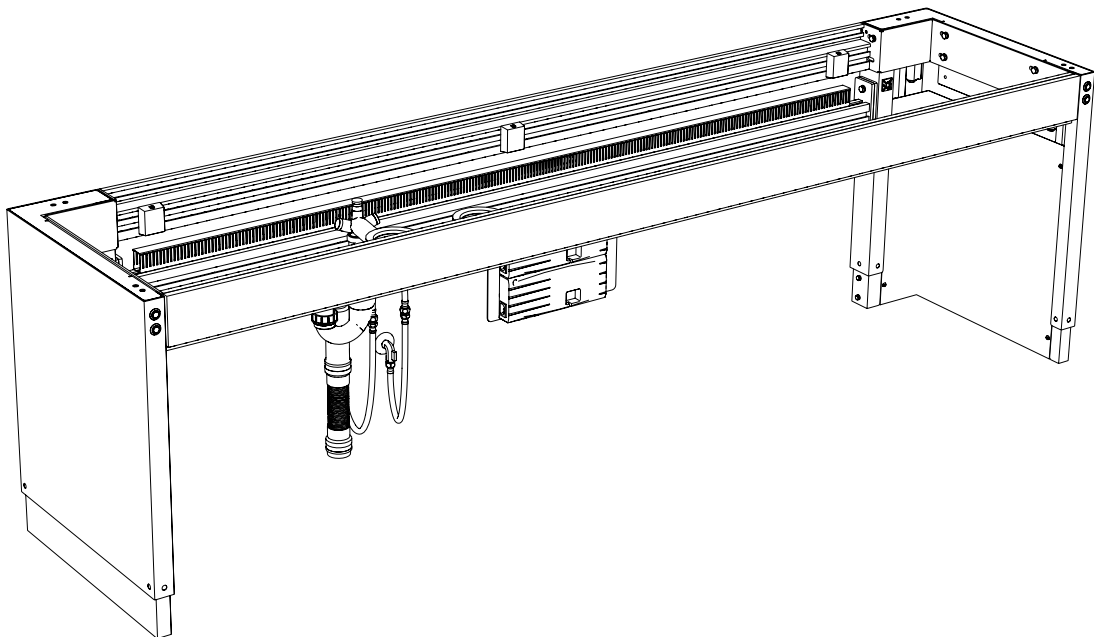
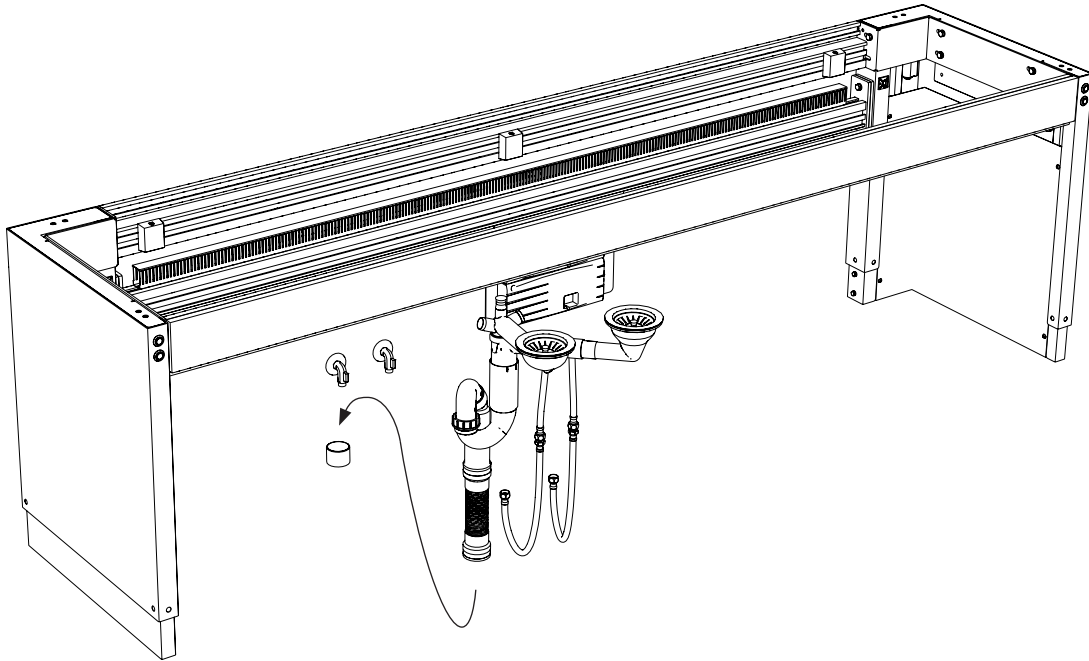
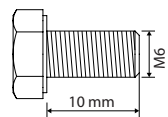
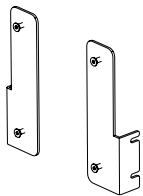
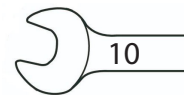
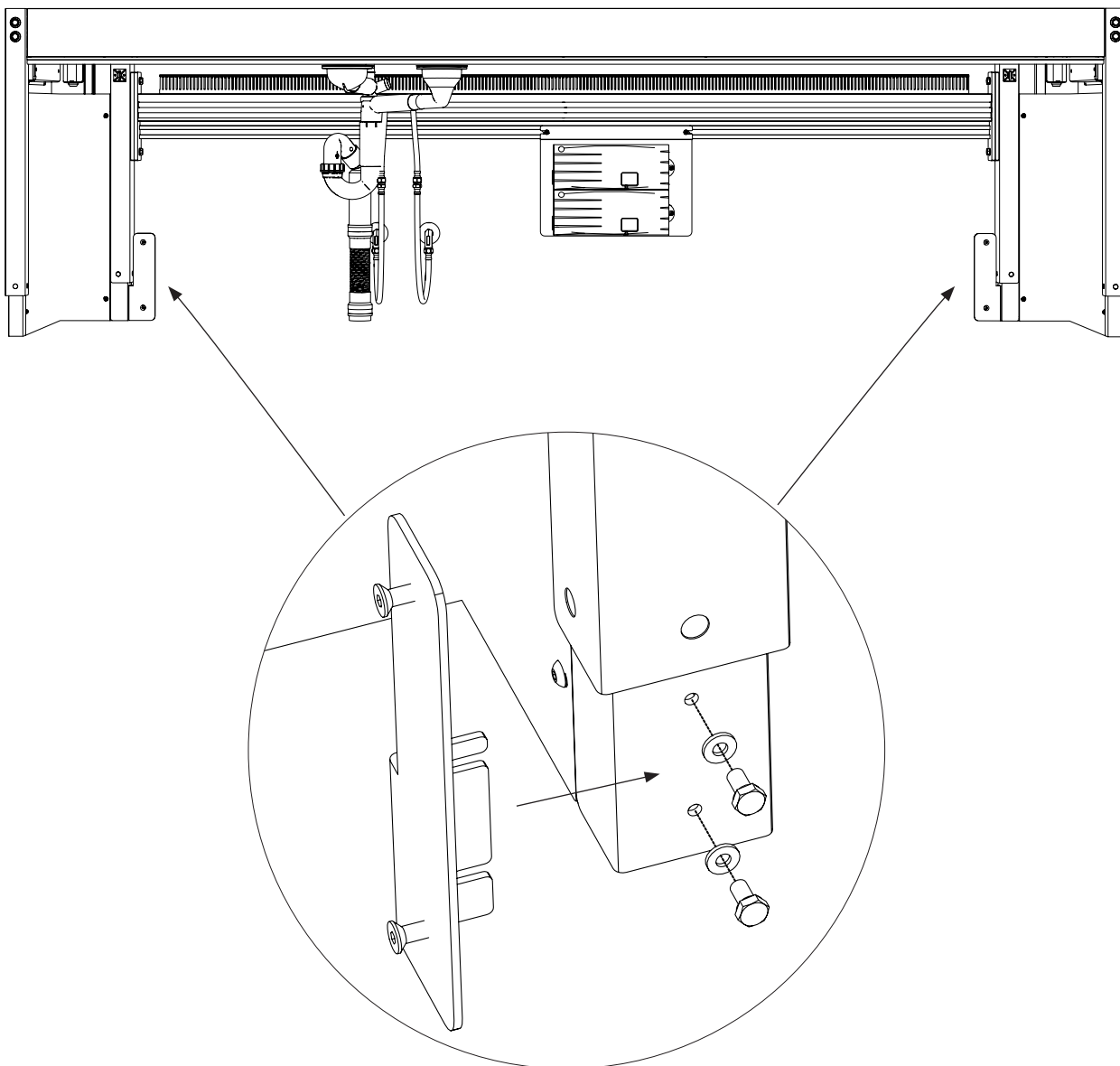
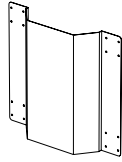
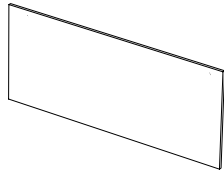
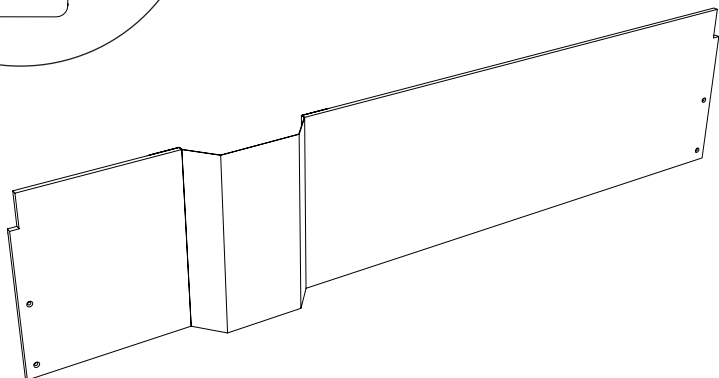
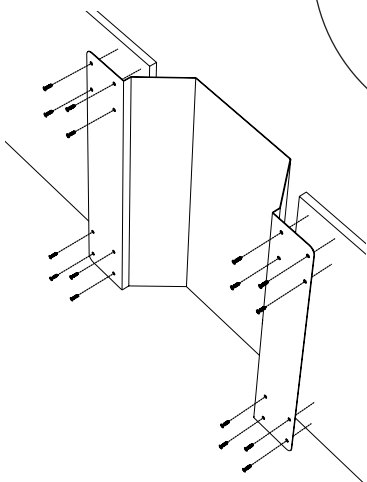
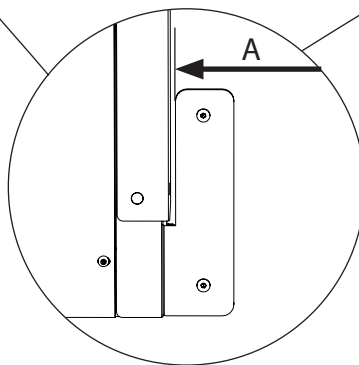
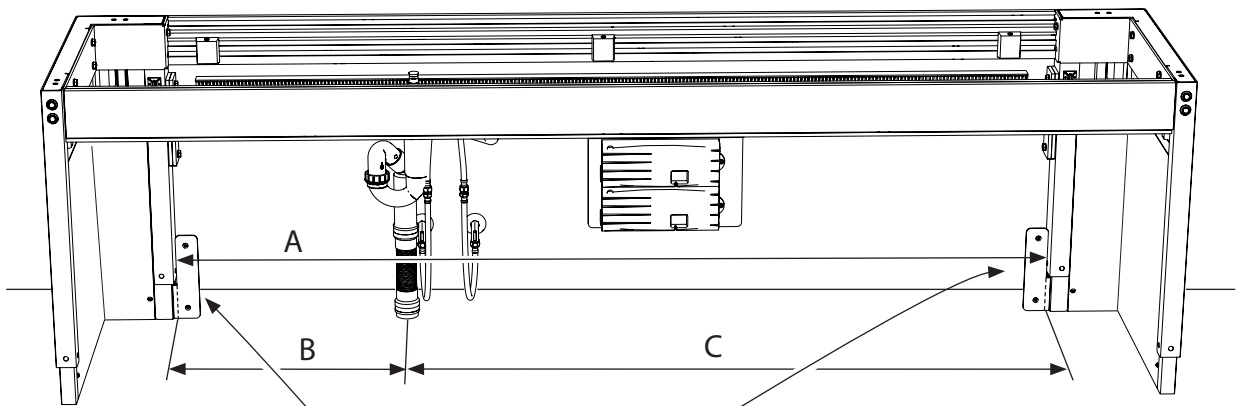
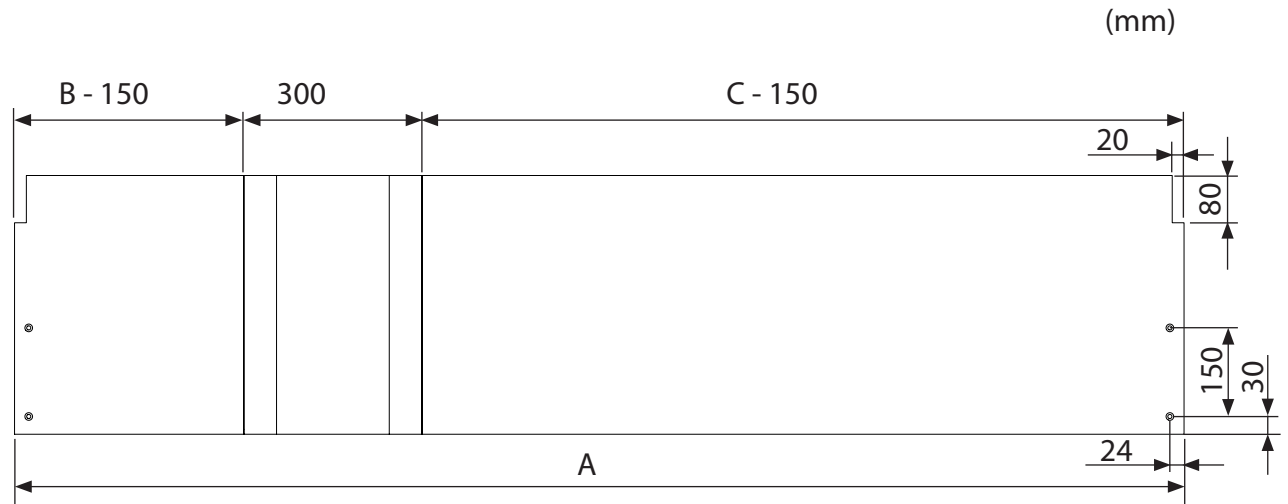
EN Cut and prepare the covers as shown below.

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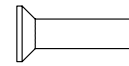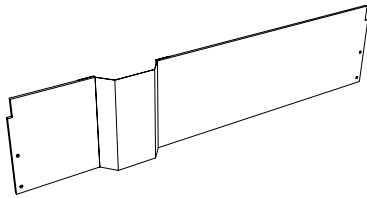
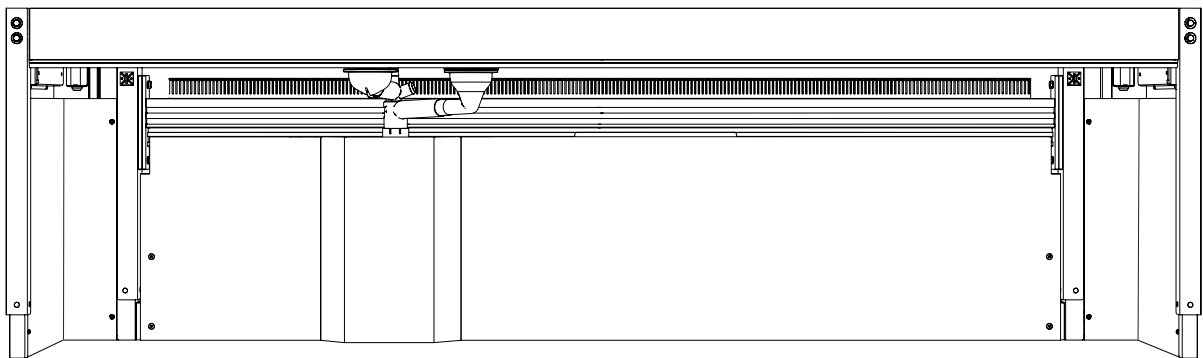
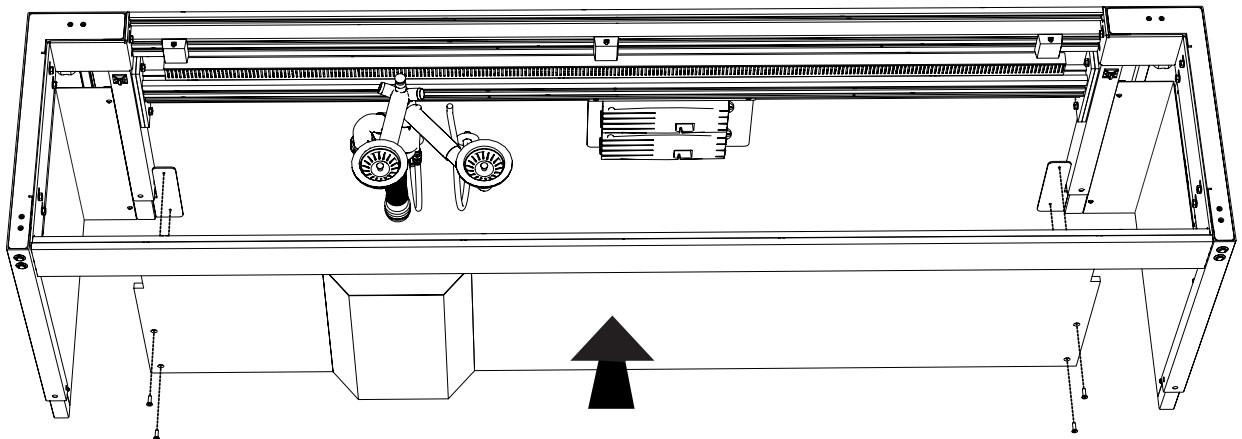
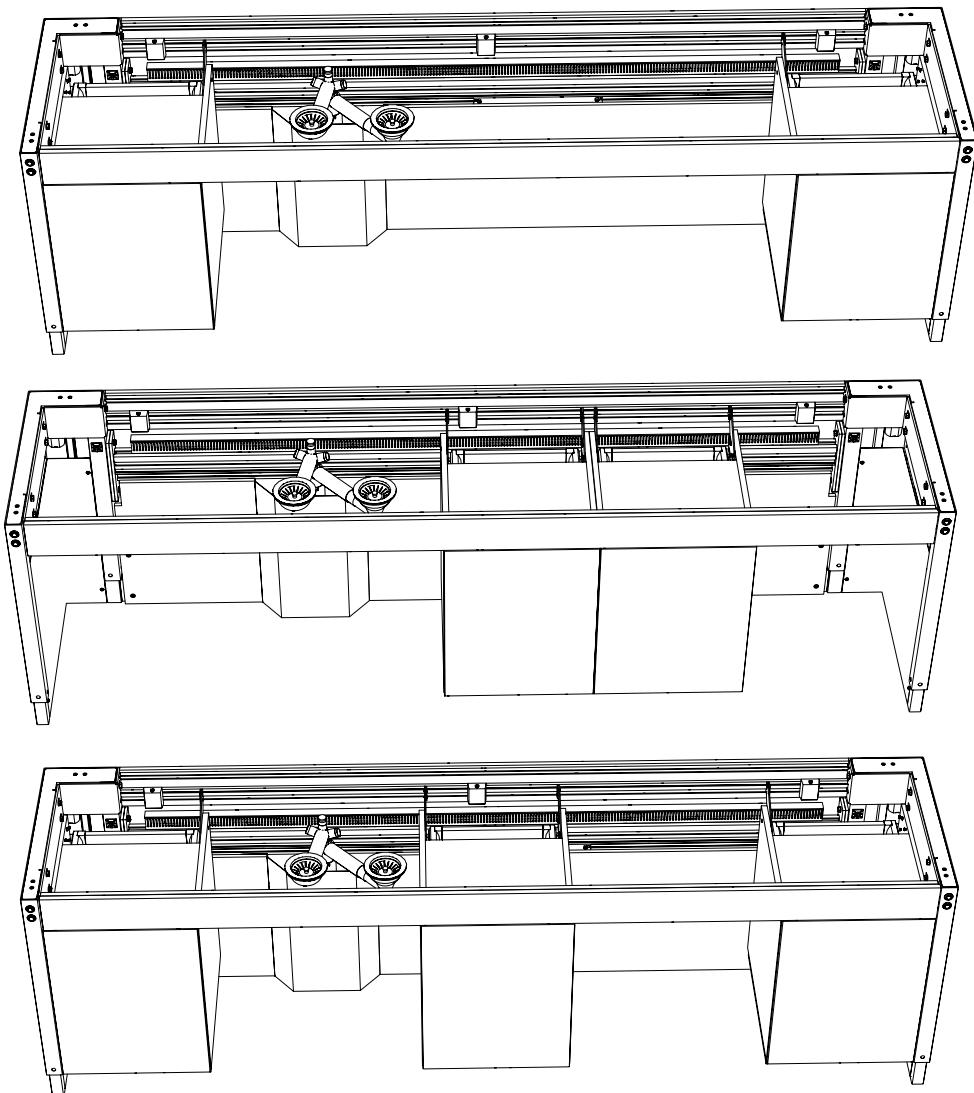
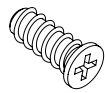
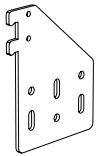
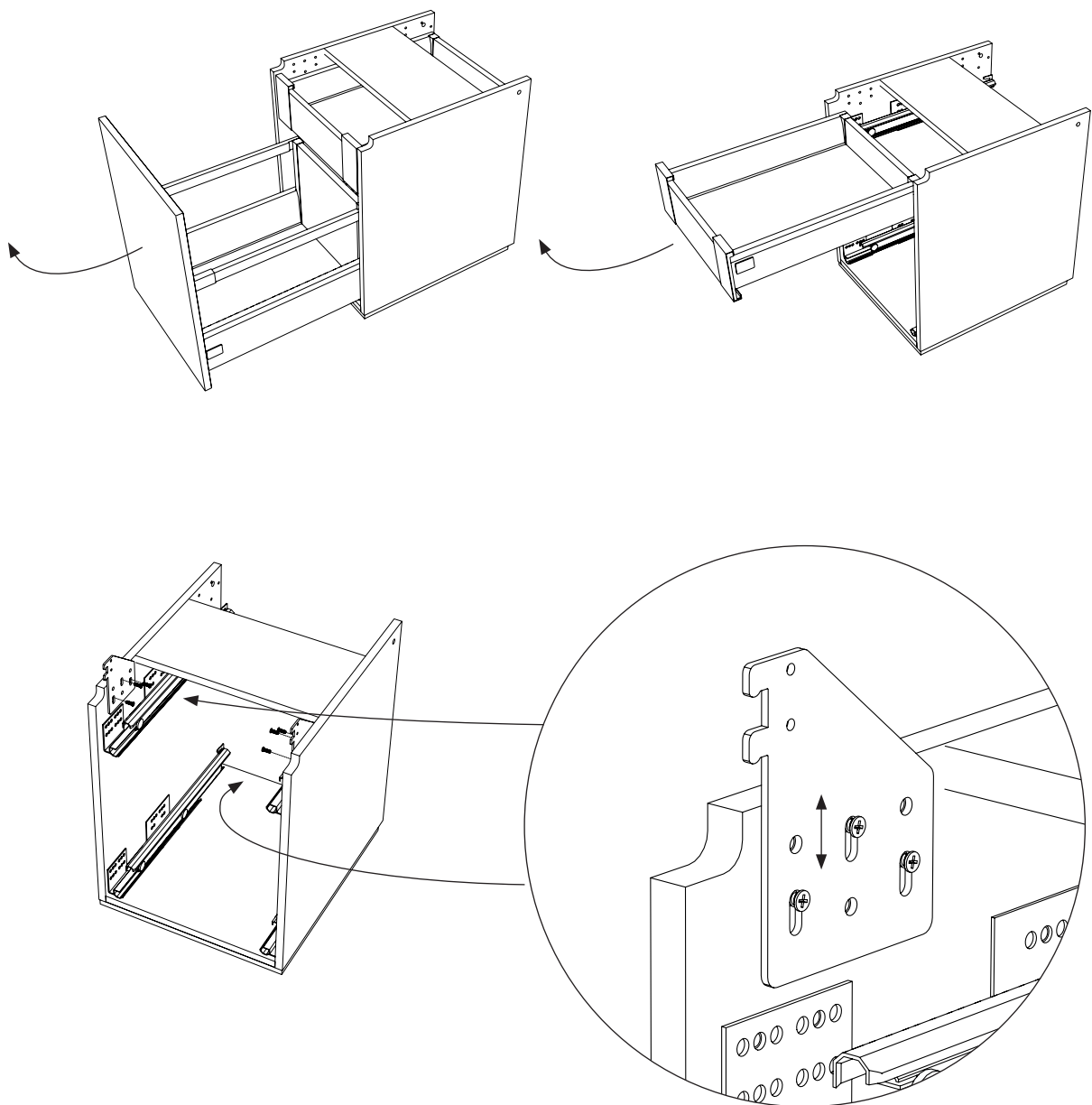
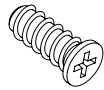
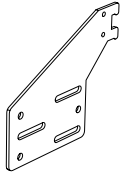
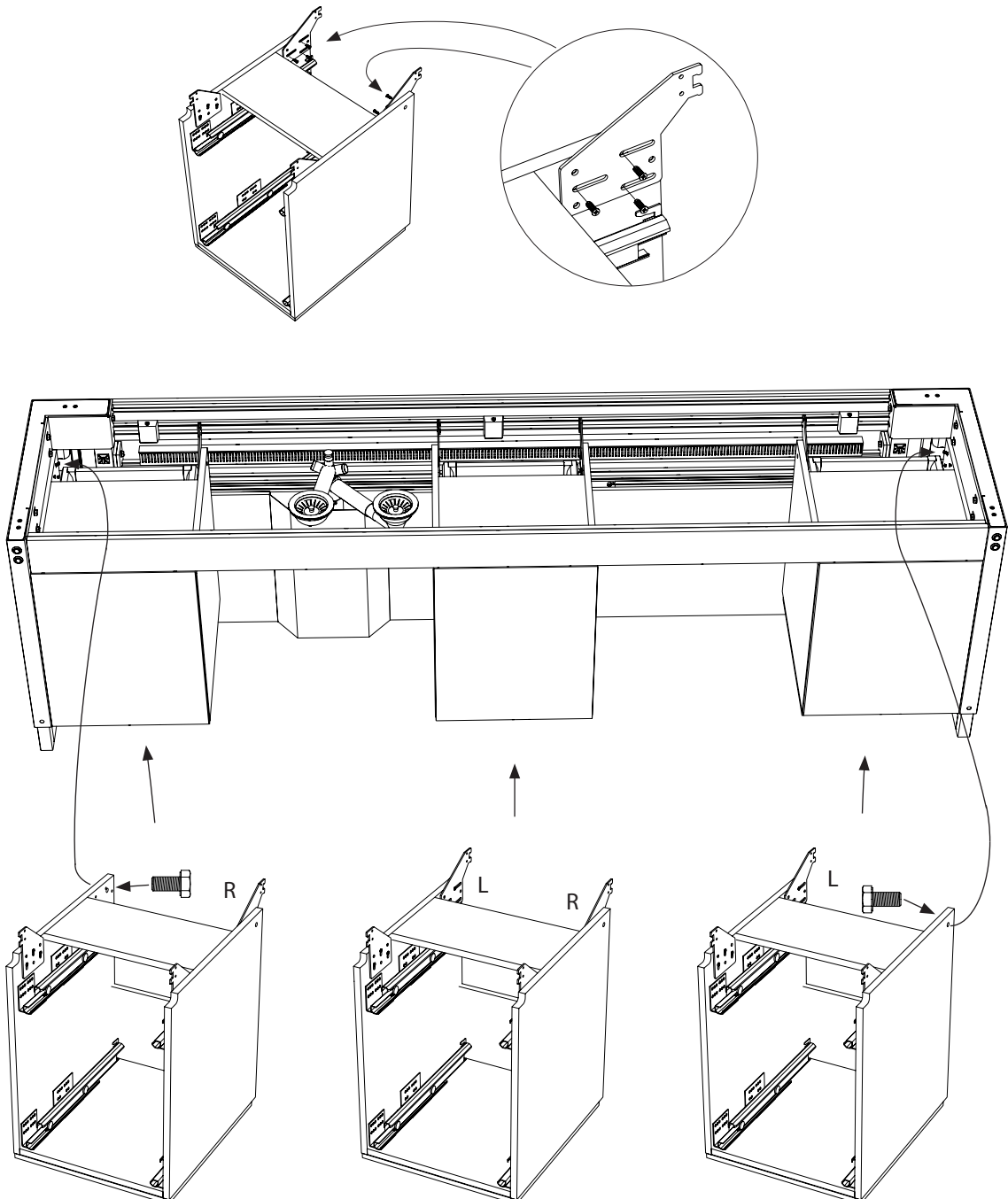
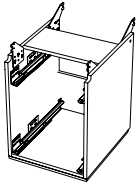
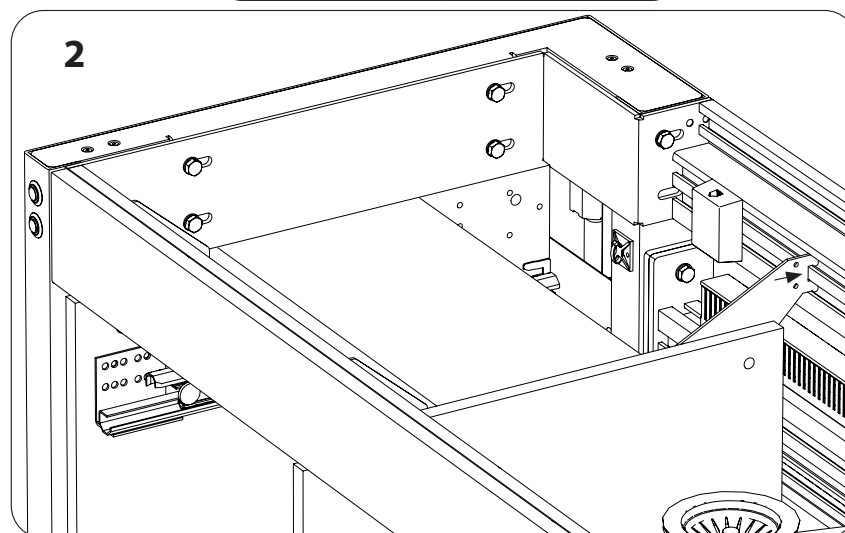
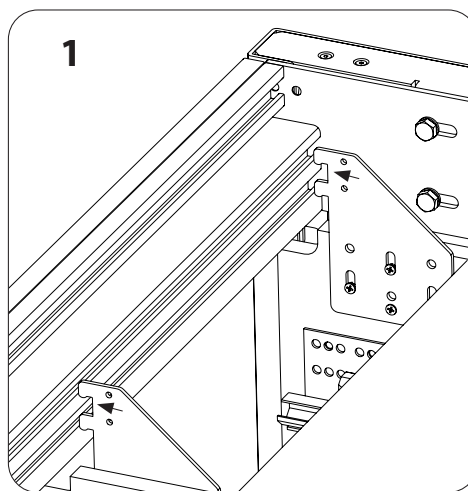
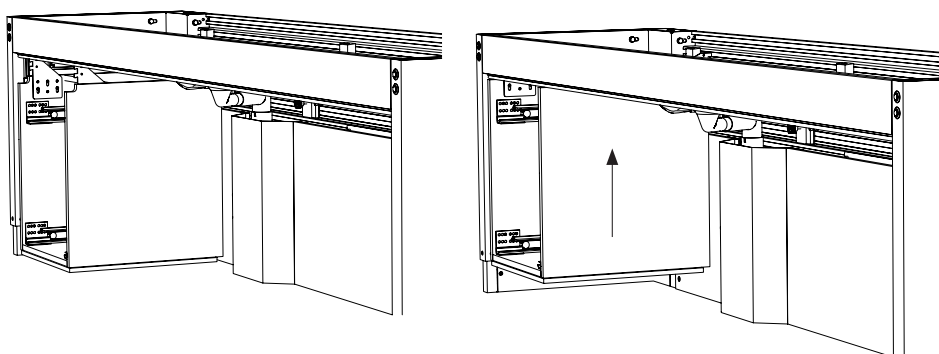
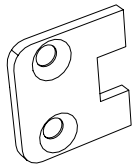
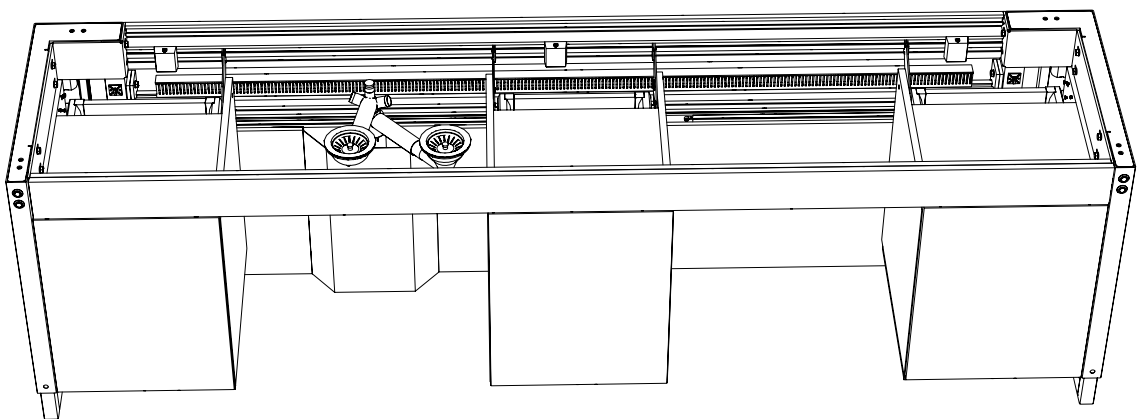
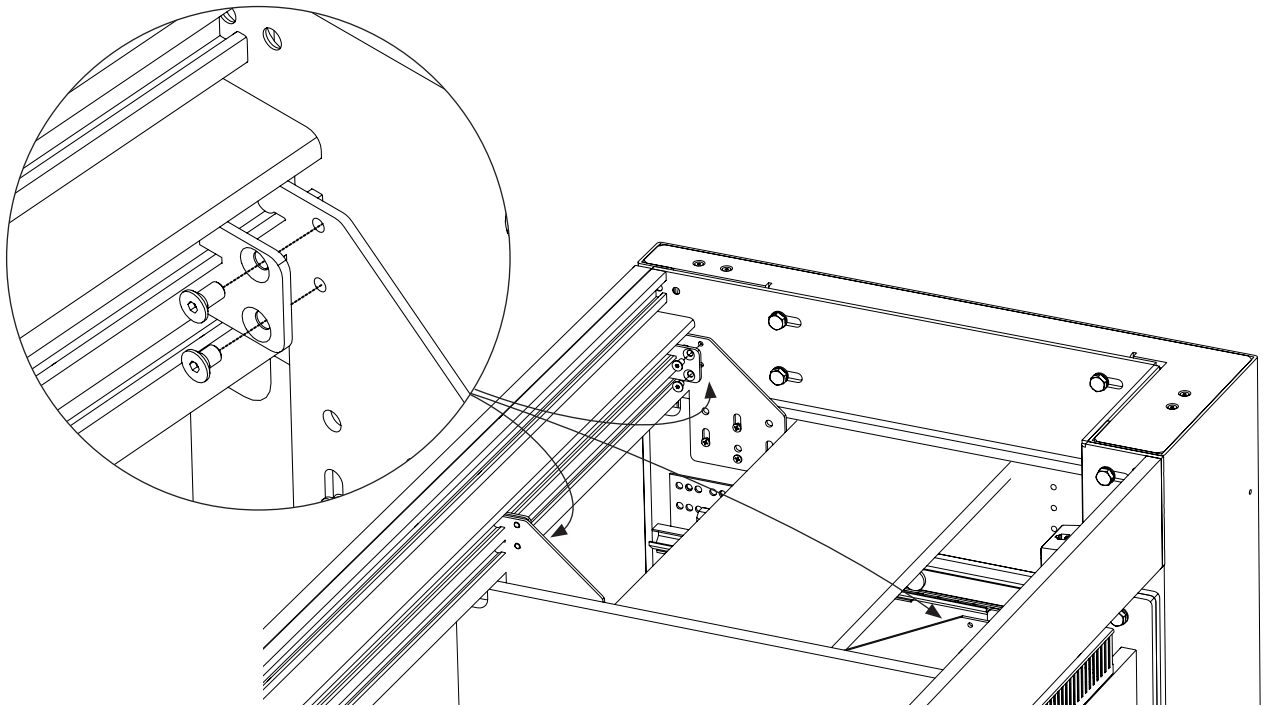
EN Screw the lock plates onto all hanger brackets.

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
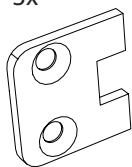
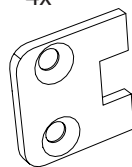
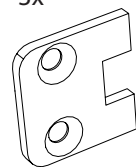
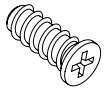
EN Now lock the height with the remaining system screws.
Lift the cabinet up to max and screw the remaining screws into all front and rear hanger brackets.

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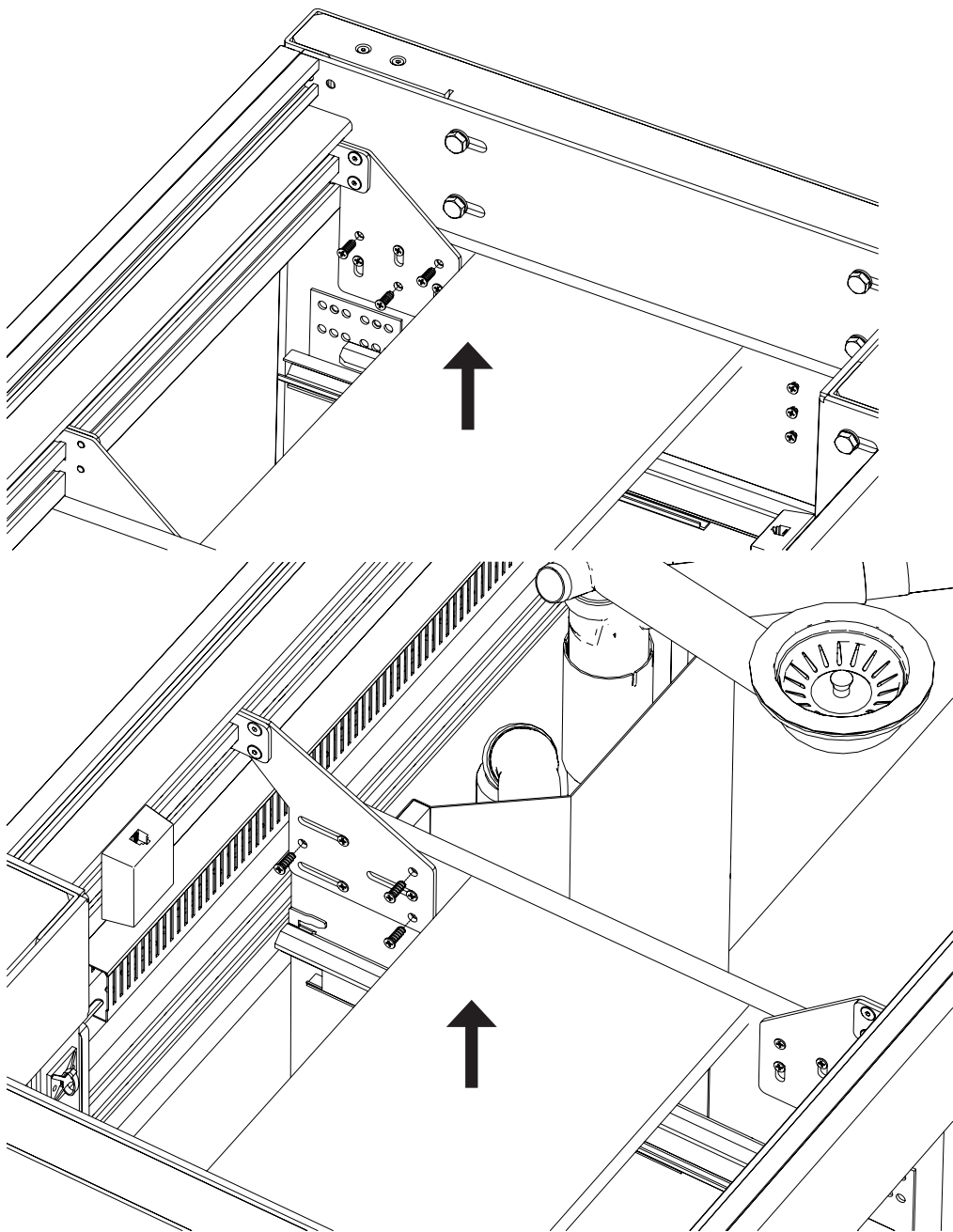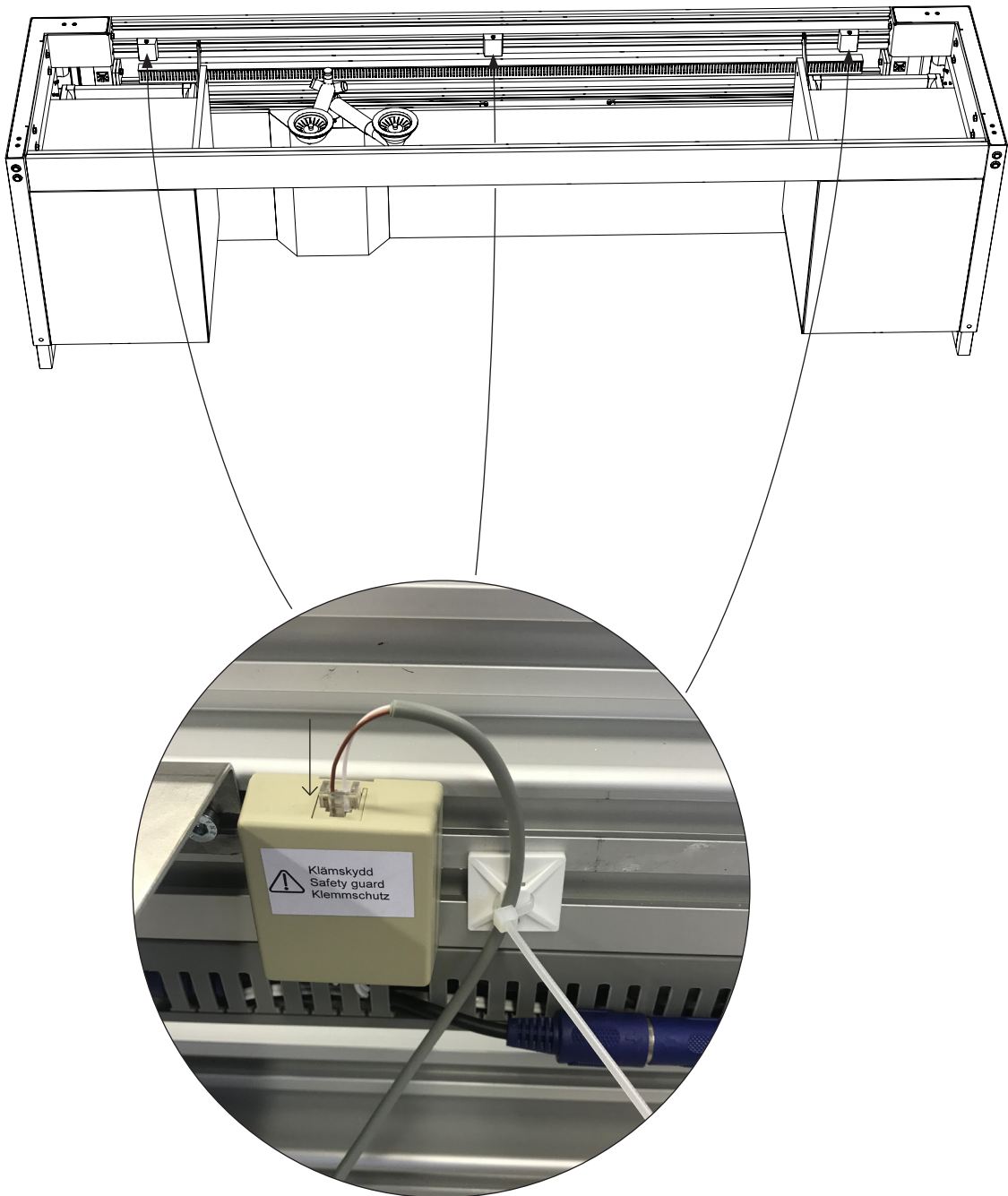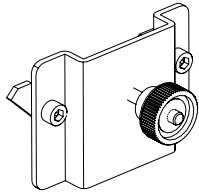
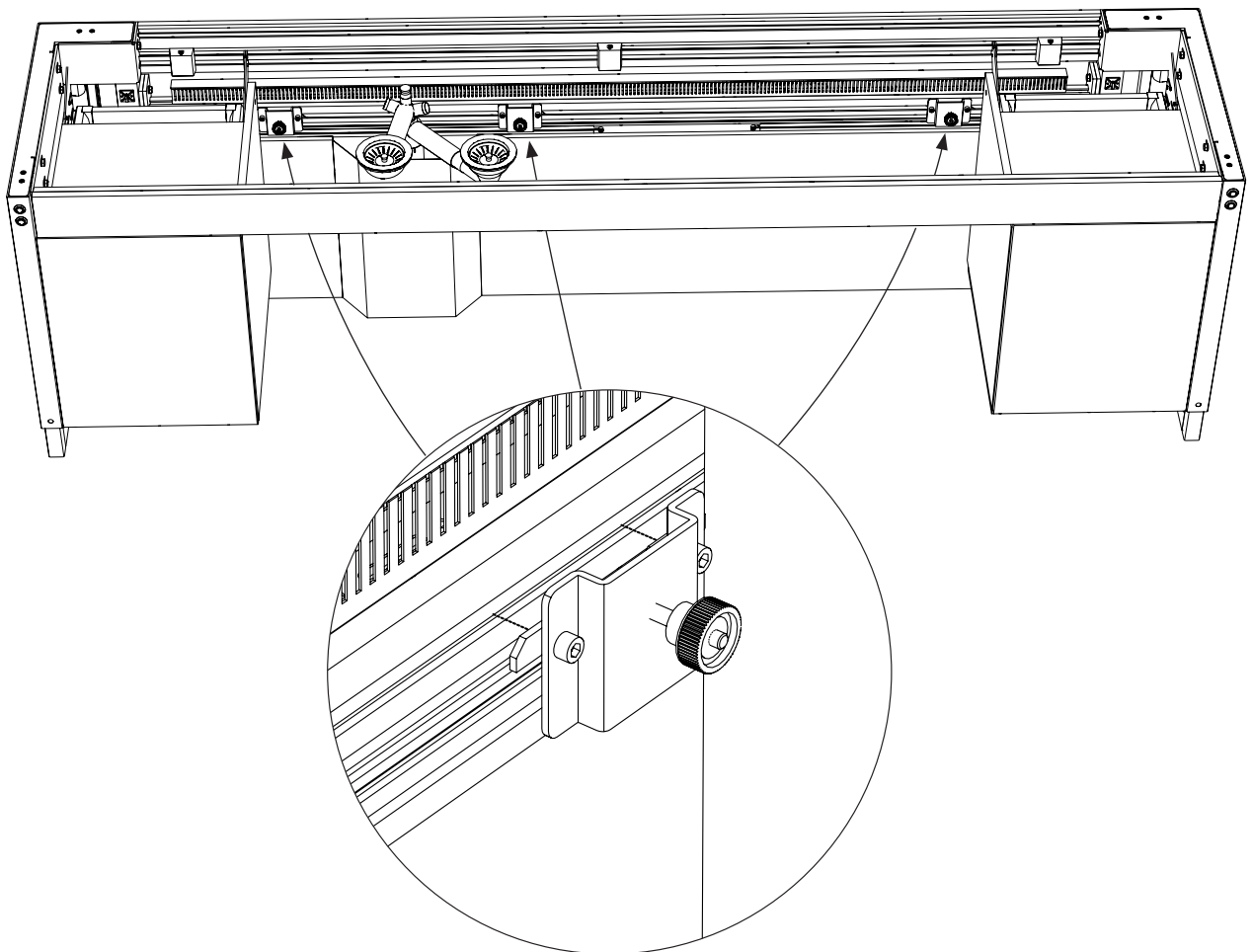
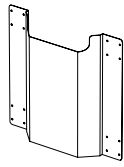
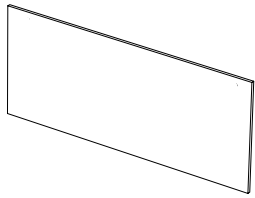
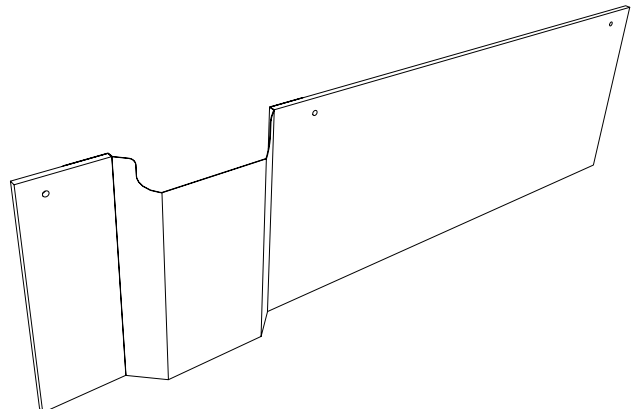
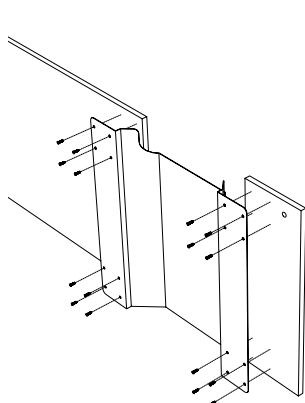
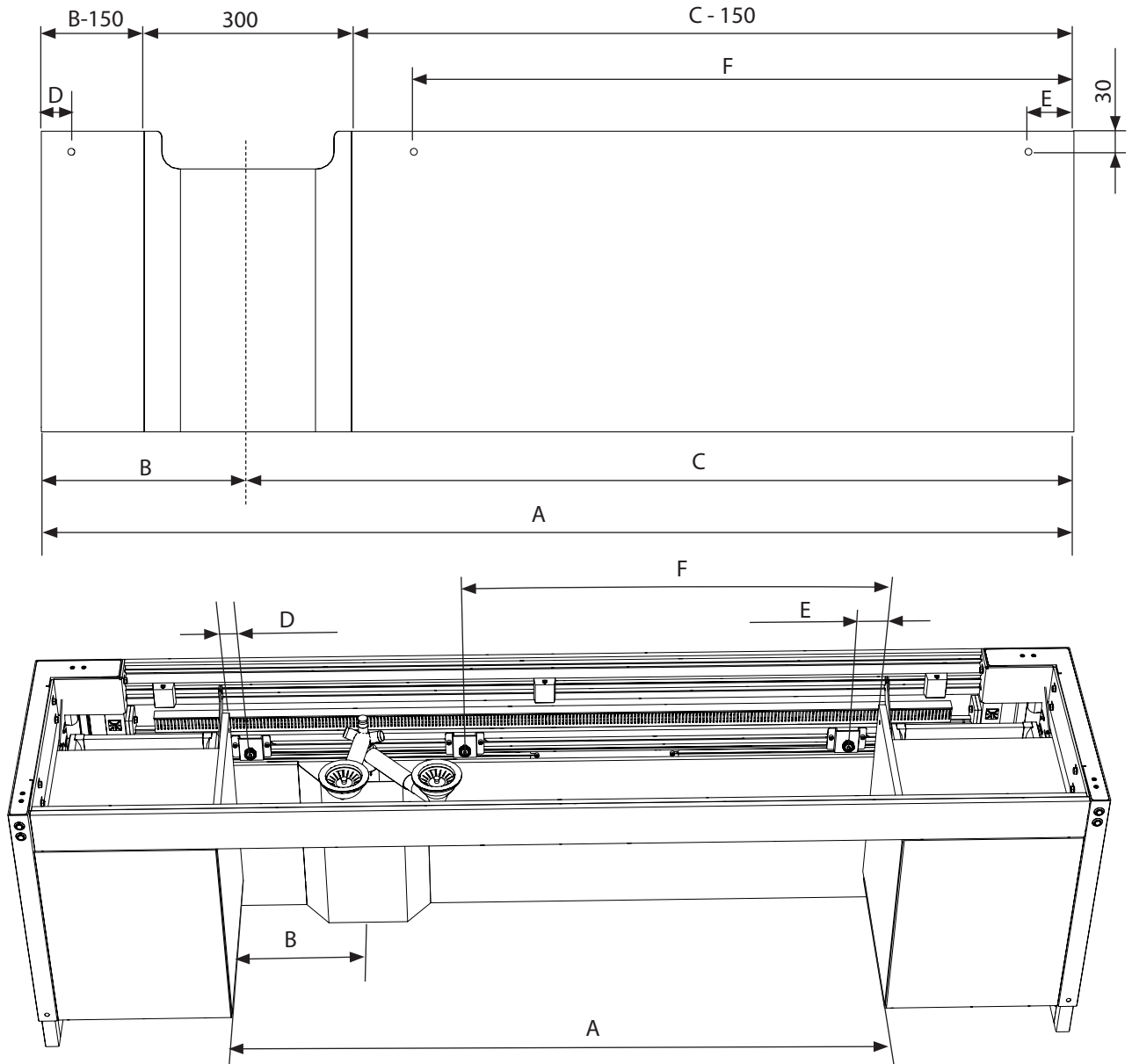
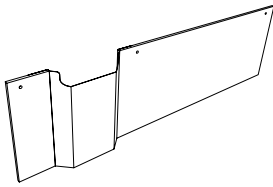
Cut and prepare the covers as shown below.

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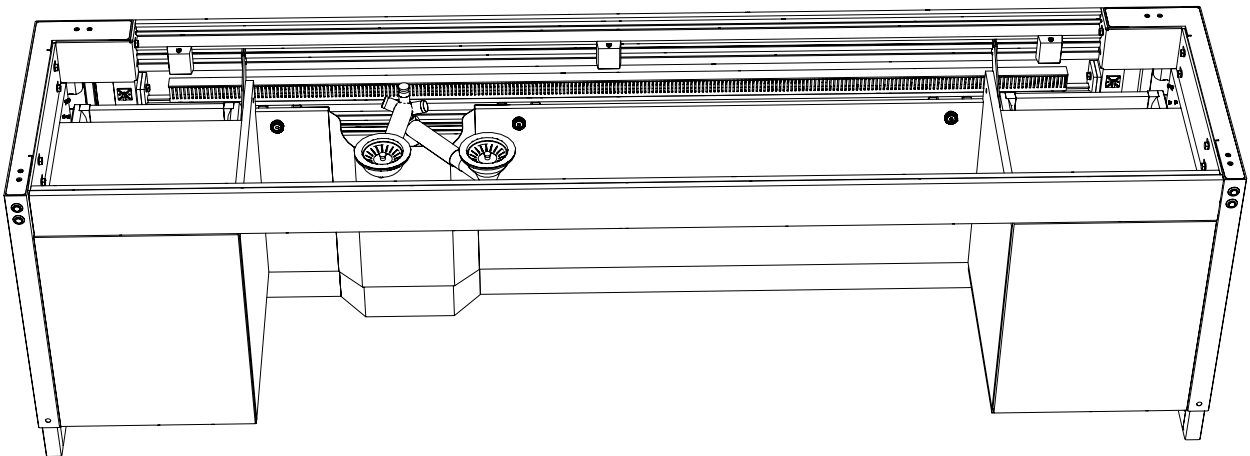
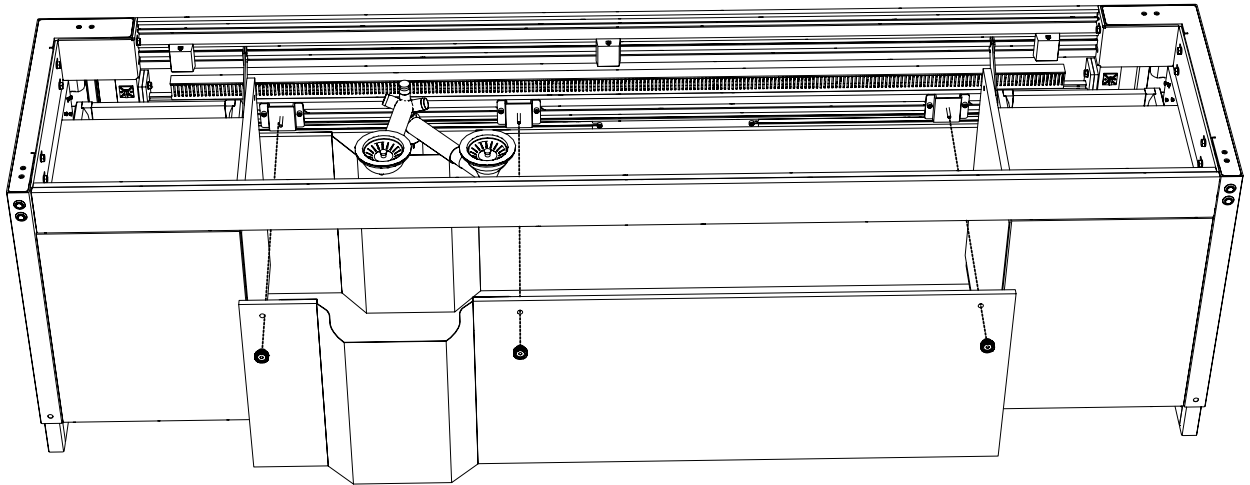
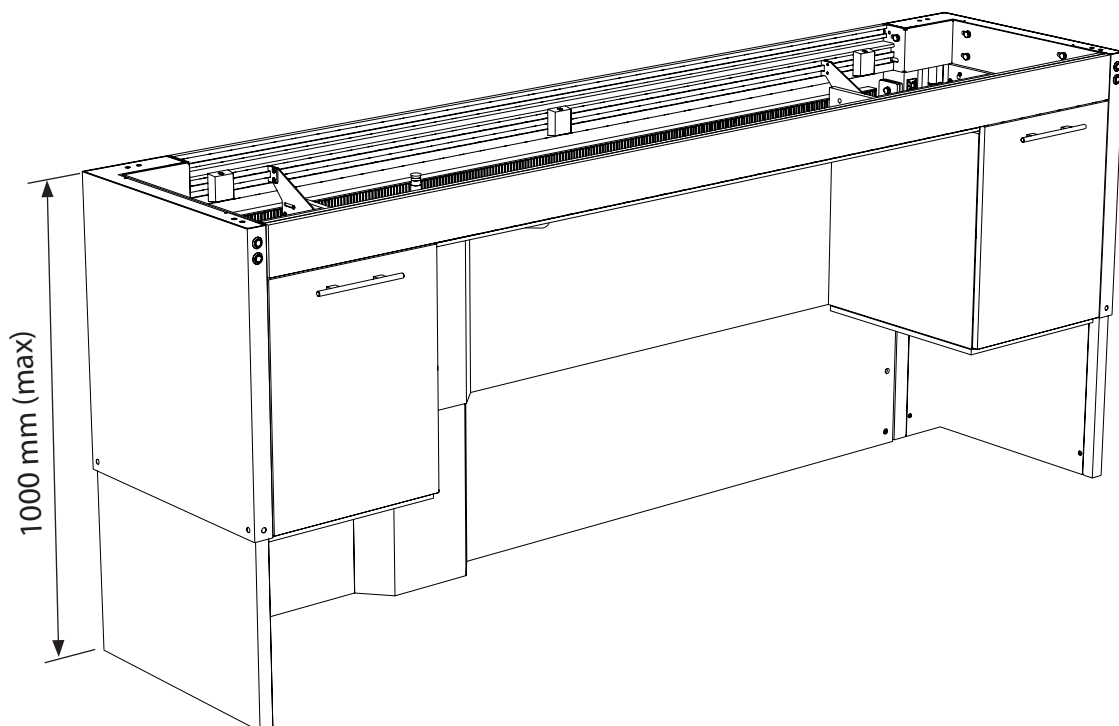
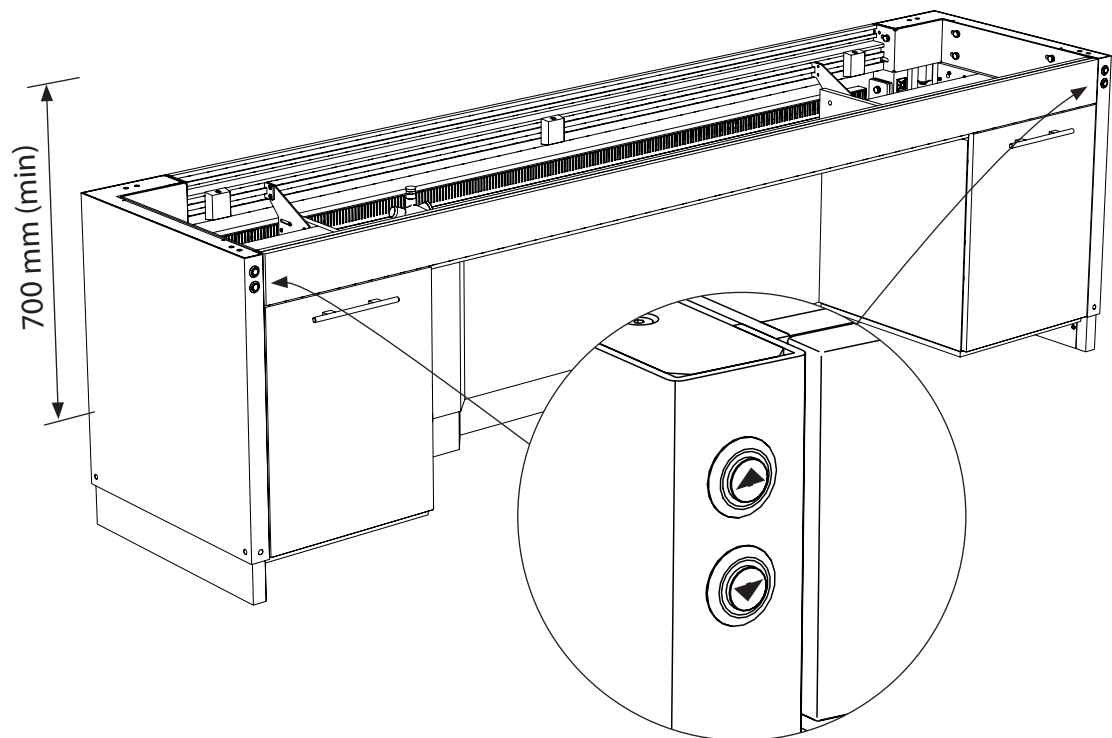
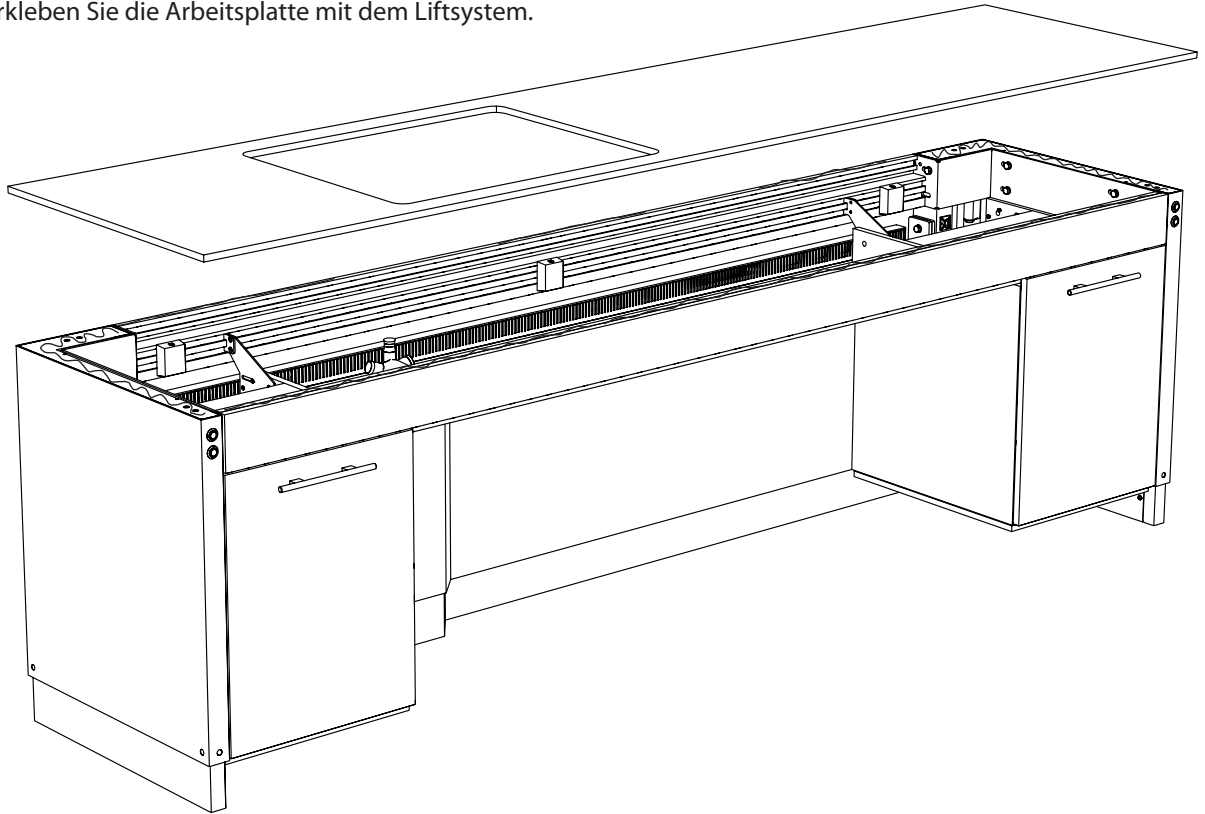
Complete functional tests should be carried out after installation:

- * Test run the lift unit up and down all the way to each end position. Make sure that the lift runs freely without obstacles, risk of pinching and noise. Make sure it remains at each end position.
- * Make sure that no cables or hoses get pinched and that they are fixed.
- * Lower the lift and make sure that the anti-pinch protection under the suspended cabinets works by pressing lightly on one at a time when lowering. Function test also anti-pinch strips under the wheelchair accessible area. Press on it while lowering. The lift should stop immediately and rise to free up space when anti-pinch protection is activated.
- * Make sure that the lift device is fixed properly on the floor.

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EN Sidelift 6400 is adjusted variably to the desired height by holding in the up or down button until the desired height is reached.
The lift can be adjusted to 700 - 1000 mm from the underside of the worktop to floor.

Release the button to stop movement.
The motors move only when a button has been pressed and held in.

Sidelift 6400 is equipped with several safety systems.
There are proximity plates under the suspended cabinets that stop the movement if obstructed.
Anti-pinch strips are fitted under the entire wheelchair accessible area that detect resistance and stop the motors.

It is important that these safety systems are not used to limit the total movement.
Nothing should be placed under the lifter that prevents it from coping with its entire movement.

Even though the lift is equipped with safety system, there is still a risk that pinching can occur.

The lift must never be operated when drawers are pulled out or if people/animals/objects are under any of the structural parts.
Read the Safe Use section on the next page.

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Safe use

The bench lift is designed and equipped with safety devices to prevent and avoid injuries and accidents.

It is still very important that the user is knowledgeable in installing and using the worktop lift.

Worktop lift in the following refers to both the machine and associated accessories such as cabinets, sink units and cooker hobs.

- Use the worktop lift only for what it is designed to do.
- It must be used in such a way that there is no risk of damage to property or personal injury.
- The worktop lift may only be used by persons who have read and understood these instructions and have permission to use it.
- Keep in mind that you as the user are responsible for the safety of others.
- The worktop lift and the work area must be in good condition. The worktop lift may not be used if damage or errors arise that affect its operational and functional safety.
- Do not touch any part of the lift when operating. Press only the manoeuvre button for the desired direction of movement.
- Never insert hands, arms or other body parts or objects over or under the worktop or cabinets when the worktop lift is being operated. There is an exceptional crush risk when wheelchair users are sitting under the worktop.
- Do not operate the worktop lift with extended drawers or open cabinet doors.
- Do not raise the worktop lift unless the space above it is free from persons or objects. Especially if the worktop lift is used under a wall cabinet lift that is adjustable in height, there is a particularly high risk that you cannot be observant of equipment or goods that may become trapped.
- Note the pinch risk between the worktop lift's top and adjacent worktops.
- Devices under the counter, such as deep sinks, can pose an increased crush risk.
- Only loads that are stable and arranged safely may be handled. Keep in mind the risk of objects left on the counter that are unstable, such as have been carelessly stacked.
- Safety devices must not be disabled or removed.
- It is forbidden to make changes to the worktop lift that affect operational or functional safety.
- Do not overload the worktop lift and ensure that the load distribution is correct.
- It is absolutely forbidden for persons to be on the bench, to step on open drawers and cutting boards or to use the worktop lift for lifting persons.
- Always strive to distribute the load over the counter to avoid an uneven load. Consider the risk that the load may start to roll, slide or fall.
- Use safe and adequate lifting methods when moving loads to and from the worktop lift. Give special attention to the handling of heavy goods, as well as goods with hazardous content or with sharp corners and edges.
- Keep in mind the risks of any work being carried out on the worktop lift. The risks depend on the equipment that is on the worktop.
- The worktop lift must undergo an annual inspection in order to prevent accidents.
- The regulations of the National Electrical Safety Board, the Swedish Work Environment Authority and other authorities must be complied with.
- Do not use the worktop lift as a lifting jack, for example, to lift goods from the floor.
- Do not use the worktop lift in a potentially explosive environment.
- When used in a public environment, especially where children can enter the machine's working area, the operator shall take sufficient measures to prevent people from entering the risk area while the machine is being operated, e.g. by cordoning off the working area.
- Signs and markings may not be removed or rendered illegible.
- Loads must not be present on the worktop or in drawers during inspections, service and repairs. The worktop lift may require dismantling to carry out service and repairs. However, the most common functions can be accessed by dismantling the cover plates.
- Installation, service and repairs may only be carried out by personnel with the required expertise.
- Only Granberg original spare parts may be used when replacing parts. Our guarantee commitment can otherwise be revoked in its entirety.
- Never change or modify any components that can in any way affect the safety and stability of the machine.
- Drawers must not be used as steps or exposed to body weight.

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Granberg Interior AB
Granberg Deutschland GmbH